- Ausgabe 2, Revision 1, 07.02.2019 -

---

# ***Afiscooter C***

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## HERSTELLER

### **AFIKIM ELECTRIC VEHICLES**

Kibbutz Afikim, 15148, Israel

Tel.: 972-4-6756451; Fax: 972-4-6751456

## WARNUNG

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Phone: 972-4-6756451; Fax: 972-4-6751456

**WARNING**

Read the operating instructions carefully before using the **AFISCOOTER C**. If you do not fully understand any part of these instructions, please contact your dealer or one of our customer service centers. Read these instructions step by step, as incorrect operation may result in injuries or damage can occur!

**WARNING**

Electromagnetic interference (EMI) can cause electric vehicles to behave unpredictably, which can be dangerous for the user. For your safety and protection, please take the time to read chapter 10 (WARNING OF ELECTROMAGNETIC FAILURES) before putting the **AFISCOOTER C** into operation.

The serial number and manufacturer's data are attached to the metal plate on the front of the **AFISCOOTER C** on the plastic mud flap (see Figure 1, page 21).

**WARRANTY**

**AFIKIM ELECTRIC VEHICLES** is constantly striving to improve its products, their quality and reliability in order to ensure both the satisfaction of its customers and its market position. The warranty for the **AFISCOOTER C** depends on the warranty provided by your local dealer.

**INFORMATION**

Only licensed sales partners of **AFIKIM ELECTRIC VEHICLE** are authorized to distribute technical information about the **AFISCOOTER C**.

Further information beyond that contained in this user manual, such as technical updates, safety instructions and others, will be provided by local dealers who have up-to-date contact details for **AFIKIM ELECTRIC VEHICLES**.

If you find any incorrect information, please contact your local dealer.

### **CE CERTIFICATION**

The registration number of Afikim Electric Vehicles with the competent authority (UK) is CA000292.

Our authorized representative with the competent authorities is:

*The Squire 12, Am Flughafen, 60549*

*Frankfurt am Main, Germany*

*[www.arazygroup.com](http://www.arazygroup.com)*

*Phone: +49 69 95932-5090*

*Fax: +49 69 95932-5200*

*E-mail: [germany@arazygroup.com](mailto:germany@arazygroup.com)*

# CONTENTS

---

<b>CHAPTER 1 - OVERVIEW OF THE AFISCOOTER C (MAIN FEATURES)</b>	<b>5</b>
<b>CHAPTER 2 - SAFETY SYMBOLS FOR THE <i>AFISCOOTER C</i></b>	<b>6</b>
<b>CHAPTER 3 - SAFETY INSTRUCTIONS for the <i>AFISCOOTER C</i></b>	<b>11</b>
<b>CHAPTER 4 - GENERAL DESCRIPTION OF THE <i>AFISCOOTER C</i></b>	<b>22</b>
<b>CHAPTER 5 - SETTING INSTRUCTIONS</b>	<b>25</b>
<b>CHAPTER 6 - OPERATING INSTRUCTIONS</b>	<b>26</b>
<b>6.1 GENERAL</b>	<b>26</b>
<b>6.2 USE OF THE LC DISPLAY</b>	<b>29</b>
<b>6.3 DRIVING AND GENERAL USE</b>	<b>31</b>
<b>CHAPTER 7 - LOADING</b>	<b>33</b>
<b>CHAPTER 8 - MAINTENANCE</b>	<b>34</b>
<b>CHAPTER 9 - STORAGE</b>	<b>38</b>
<b>CHAPTER 10 - TROUBLESHOOTING AND REPAIR</b>	<b>39</b>
<b>CHAPTER 11 - WARNING OF ELECTROMAGNETIC INTERFERENCE</b>	<b>41</b>
<b>Appendix A - Technical data</b>	<b>43</b>

## **CHAPTER 1 - OVERVIEW OF THE *AFISCOOTER C* (MAIN FEATURES)**

---

Congratulations on choosing the ***AFISCOOTER C*** as your electric scooter.

The ***AFISCOOTER C*** is an electric scooter for senior citizens and people with disabilities as well as for leisure and sport.

Electric scooter classification: Class B according to EN12184.

The ***AFISCOOTER C*** is designed for a maximum payload of up to 135 kg (3-wheel version) or 150 kg (4-wheel version).

The ***AFISCOOTER C*** is designed for senior citizens and people with disabilities. A certain level of motor skills, vision and cognitive ability is required to operate the scooter. It is recommended that you consult your doctor to assess your ability to operate the scooter. The ***AFISCOOTER C*** is an electric scooter for indoor and outdoor use and is designed for normal ambient conditions from very cold temperatures (-30 °C) to very hot temperatures (+45 °C).

The ***AFISCOOTER C*** also works in light rain showers, but prolonged use in heavy rain or snow is not recommended.

The ***AFISCOOTER C*** is easy to operate. The relatively large wheels make it possible to drive on unpaved roads, over moderately rough terrain and over obstacles up to a height of 12 cm.

Appendix A "Data sheet" (page 30) contains information on the required tire pressure.

The maximum angle of incline that the ***AFISCOOTER C*** can safely negotiate is: 9° (16%). Do not attempt to negotiate inclines and declines that are steeper than the limits of the scooter.

The braking system stops the vehicle gently and within a short distance after the drive lever is released.

The ***AFISCOOTER C*** is equipped with shock absorbers at both the front and rear to provide the user with a comfortable and smooth ride, even when the vehicle is used on moderately rough roads.

A normal handbrake is also available as a reserve.

The seat is spacious and comfortable and can be rotated for easy access (standard seats). It can be adjusted forwards and backwards. The seat can also be adjusted up and down by a technician.

All controls are conveniently located on the steering column and are easy to operate.

Under normal conditions, the ***AFISCOOTER C*** can travel approx. 40 km before the batteries need to be recharged. The distance is reduced if the scooter is frequently used on inclines, uneven terrain or to negotiate kerbs, etc.

After reading through this entire manual and before using the ***AFISCOOTER C***, please carry out a brief visual inspection of the scooter to ensure that there is no visible damage. If you have any questions or concerns, please contact your dealer.

The ***AFISCOOTER C*** was developed exclusively for the use of maintenance-free, leak-proof batteries.

**Maintenance instructions are available for trained technicians.**

**For visually impaired people, a digital format of the instructions for use can be downloaded from our website [www.afiscooters.com](http://www.afiscooters.com)**

## CHAPTER 2 - SAFETY SYMBOLS FOR THE *AFISCOOTER C*

---

**WARNING!** An authorized supplier or qualified technician must carry out the initial setup of this scooter and perform all procedures in this manual.

The following symbols are used in this instruction manual and on the scooter to indicate warnings and important information. It is very important that you read and fully understand them.

**WARNING!** Indicates a potentially hazardous condition or situation. Failure to follow the prescribed procedures may result in injury, damage to components or malfunction. This is indicated on the scooter by a triangle with a black border.

**COMMANDMENT!** These actions must be carried out as specified. Failure to carry out the prescribed actions may result in injury and/or damage to the device. This is indicated on the scooter by a square with a white border on a dark background.

**PROHIBITED!** These actions are prohibited. These actions must not be carried out at any time and under any circumstances. Performing a prohibited action may result in injury and/or damage to the device. This is indicated on the scooter by a black circle with a slash.

Please observe the following safety instructions and adhere to them at all times. Additional warnings and instructions are printed in this user manual. Be sure to read all chapters of this manual carefully before operating the *AFISCOOTER C*. It is also advisable to refresh your memory by re-reading these instructions from time to time.

Familiarize yourself with the *AFISCOOTER C* and its controls before riding. Ride slowly (speed limiter set to MIN) until you have familiarized yourself with the *AFISCOOTER C*, its various functions, safety features, braking options and limits.

Watch out carefully for pedestrians and ride the *AFISCOOTER C* accordingly.

Always ride at half speed in busy areas. Only ride where this is permitted under all applicable local laws and regulations. The *AFISCOOTER C* is 66 cm wide. Pay attention to make sure that you do not hit any persons or objects in the vicinity of the vehicle.

**Never remove safety components from the *AFISCOOTER C***, z. e.g. mud flaps, plastic covers or the emergency brake lever. **Never remove warning signs**. If one of the warning signs has been removed, worn or painted over, etc., ask your dealer to provide you with a new sign.

**Never** put your hands, fingers or legs near moving parts or under protective covers, as there are moving parts and hot surfaces under these covers.

When making adjustments (folding back the seat backrest, adjusting the armrest, moving the seat forwards or backwards, adjusting the distance to the steering column) and when operating the driving lever or handbrake, always ensure that your hands and fingers are not trapped between the moving parts.

When adjusting the steering column forwards or backwards, make sure that you do not get your fingers caught in the mechanism.

**Never** load the *AFISCOOTER C* with more than 135 kg (3-wheel version) or 150 kg (4-wheel version) (depending on model) payload (user weight and all other loads together, including baskets). The load in the rear basket must not exceed 8 kg.

The load in the front basket must not exceed 5 kg.

In the version with one seat, the *AFISCOOTER C* is designed to carry only ONE person. Never ride it with an additional person on board, not even with a child.

Never inflate the *AFISCOOTER C* tires to a pressure higher than the pressure recommended by the manufacturer and indicated on the tires.

Never use the *AFISCOOTER C* on stairs.

Never attempt to drive over an obstacle that exceeds the recommended maximum angle of inclination and the obstacle height specified in the data sheet. When driving over an obstacle, always drive so that the front of the *AFISCOOTER C* is facing the obstacle.

If for any reason the *AFISCOOTER C* does not stop when you release the drive lever or takes longer to stop than it should (3 meters on a flat surface), immediately use the handbrake to stop the *AFISCOOTER C*, turn the master key switch to OFF and contact your dealer.

Do not use the *AFISCOOTER C* in heavy rain or extreme humidity or at temperatures below -30 °C or above +45 °C.

**WARNING:** Surfaces can be hot!

The seat weighs 21 kg. Each battery weighs at least 14 kg. The drive unit weighs at least 25 kg. Be careful when lifting the seat and the batteries.

See instructions for lifting the seat and batteries in chapter 8 "Maintenance" and Chapter 9 "Troubleshooting and repair". Always lift with your knees and legs, not with your back.

If the **AFISCOOTER C** is involved in an accident in which parts are damaged, you must have it inspected by a qualified technician before continuing to use it.

Batteries contain lead and acid, which is very corrosive. Batteries must be handled and disposed of in accordance with the appropriate procedures.

They are not intended for use as a seat in a motor vehicle.

When driving, always ensure that the backrest is in its fully upright position.

Only use maintenance-free, leak-proof lead-acid batteries.

**WARNING!!!** Special care must be taken to ensure that children do not touch the

cannot operate the **AFISCOOTER C**.

Do not leave unsupervised children near the scooter

**AFISCOOTER C** play when the batteries are charged.

Always drive carefully. Adapt your speed to the road conditions.



The **AFISCOOTER C** can become unstable when changing direction at high speed. Reduce the speed to a minimum before sharp bends to prevent the scooter from tipping over.

The **AFISCOOTER C is capable** of climbing slopes that are steeper than the recommended safe gradient. However, do not ride on slopes where you do not know the angle of inclination. Always ride very slowly and do not ride sideways on such slopes.

Make sure that your emergency brake is working properly before setting off.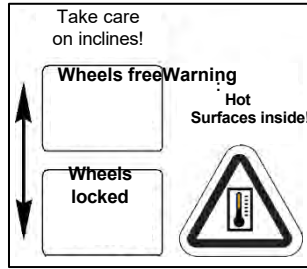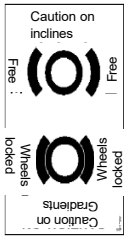

Do not release the manual parking brake or the manual release lever of the electromagnetic brake (EMB) when the **AFISCOOTER C** is on a slope.  
Never switch the **AFISCOOTER C** to freewheel mode on any type of incline while you are sitting on it or standing next to it.

The **AFISCOOTER C** lighting should be switched on when visibility is limited, day or night.

As the seat cushion is fire-resistant, you must not cover it or replace it with another cushion that is not of the same type.  
Watch out for hot parts when removing the battery cover. The motor, the axle, the electromagnetic brake (EMB), the electronic cards and the electronic control unit may be hot.  
Do not touch them until you have checked their temperature.

Make sure that your fingers are not trapped between the levers and the rubber grip.

Do not use a cell phone, walkie-talkie, laptop or other radio transmitters while riding the **AFISCOOTER C**.

Removing the grounding pin can be an electrical hazard. If it is necessary to use a 3-pin socket, properly install an approved 2-pin adapter.

Do not connect an extension cable to the rectifier or the battery charger.

Keep tools and other metal objects away from the battery terminals. Contact with tools can cause an electric shock.  
Do not sit on the **AFISCOOTER C** when it is on a hoist or lifting device.

Explosion hazard

Disposal and recycling: The **AFISCOOTER C** is made of recyclable materials (plastic parts, batteries, electronics, etc.). Information on the proper disposal of your **AFISCOOTER C**, its parts and packaging can be obtained from your authorized specialist dealer.

Wear safety goggles


Contains lead

The battery charger is only suitable for indoor use.

Flammable material! Keep away from naked flames.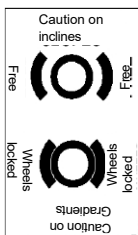

The manual release lever for the electromagnetic brake (EMB) is located at the rear of the **AFISCOOTER C**. When using this lever, make sure that you do not touch the surface of the motor, as it can be very hot and cause injury. Observe the warning sign on the back of the scooter near the lever.

## CHAPTER 3 - SAFETY INSTRUCTIONS for the *AFISCOOTER C*

---

### GENERAL

**DANGER!** Read and understand these operating instructions completely before riding your new *AFISCOOTER C* scooter for the first time.

Your *AFISCOOTER C* is a modern device for increasing quality of life and mobility. We offer an extensive range of products

that best meets the individual needs of the user. Please note that the final selection and purchase decision on the type of *AFISCOOTER C* scooter is the responsibility of the user who is capable of making such a decision, as well as the responsibility of their healthcare professional (i.e. doctor, physiotherapist, etc.).

The content of this manual assumes that a mobility aid professional has adapted the *AFISCOOTER C* mobility scooter to the user and has assisted the prescribing healthcare professional and/or authorized dealer in providing instructions for the use of the product.

There are certain situations, including some medical conditions, in which the user must practice operating the *AFISCOOTER C* mobility scooter in the presence of a trained caregiver. A trained attendant can be defined as a family member or professional who is specially trained to assist a user with various activities of daily living.

When you start using your *AFISCOOTER C* scooter in your daily activities, you will probably encounter situations where you need some practice. Just take your time and you will soon be in full and safe control when driving through doorways, in and out of elevators, up and down ramps and maneuvering over moderately uneven terrain.

Below you will find some precautions, tips and other safety considerations to help you ride the *AFISCOOTER C* safely.

### MODIFICATIONS

We have developed and designed the *AFISCOOTER C* to offer you maximum mobility and utility. Under no circumstances should you use any feature, part or function of your

Change, add, remove or deactivate *AFISCOOTER C*. **WARNING!** Do not modify your *AFISCOOTER C* in any way that is not authorized by the manufacturer. Do not use any accessories that have not been tested or approved by the manufacturer.

### REMOVABLE PARTS

**WARNING!** Do not attempt to lift or move your *AFISCOOTER C* by any of its removable parts, including the armrests, seat or cover.

## SQUASHING POINTS

There are several points in the **AFISCOOTER C** that could potentially be pinch points. This mainly relates to moving parts such as wheels, drive levers, etc. Always keep your hands, fingers or legs away from moving parts.

## SAFETY CHECK BEFORE DEPARTURE

Get to know the feel of your **AFISCOOTER C** and its capabilities. We recommend that you carry out a safety check before each use to ensure that your **AFISCOOTER C** functions smoothly and safely. Carry out the following inspections before you use your scooter

- ✓ Check the correct tire pressure. Maintain the air pressure specified on each tire, but do not exceed it.
- ✓ Check all electrical connections. Make sure that they are secure and not corroded.
- ✓ Check all connections of the wiring harness. Make sure that they are properly fastened.
- ✓ Check the brakes.
- ✓ Check the charge status of the battery.
- ✓ If you notice a problem, contact your authorized **AFISCOOTER C** dealer for assistance.

## TIRE PRESSURE

Your **AFISCOOTER C** is equipped with pneumatic tires. Check the tire pressure at least once a week. The correct tire pressure extends the service life of your tires and helps to ensure the smooth operation of your **AFISCOOTER C**.

**WARNING!** It is essential that the air pressure specified on each tire in psi, bar and kPa is maintained at all times. Do not under-inflate or over-inflate your tires. Under-inflation can lead to loss of control, over-inflation can cause the tire to burst. If the specified pressure is not always maintained, this can result in tire and/or wheel damage.

**WARNING!** Inflate your scooter tires from a regulated air source with an attached pressure gauge

**NOTE:** If the tires on your **AFISCOOTER C** are only marked with a psi value, use the following conversion formulas to determine the bar or kPa value:

$$\text{Bar} = \text{psi} \times 0.06895$$

$$\text{kPa} = \text{psi} \times 6.89476.$$

## WEIGHT RESTRICTIONS

Your scooter is designed for a maximum weight. Details can be found in Appendix A "Technical data".

**DUTY!** Stay below the specified payload limit for your

**AFISCOOTER C**. Exceeding the payload limit will invalidate your warranty.

We accept no responsibility for injuries and/or damage to property resulting from non-compliance with the weight restrictions.

**WARNING!** Do not carry passengers on your scooter. The transportation of passengers on your

**AFISCOOTER C** can lead to instability.

## INFORMATION ABOUT INCLINATIONS


More and more buildings have ramps with specific angles of inclination designed for easy and safe access. Some ramps may have turning loops (180-degree turns), which ensure good cornering behavior on your


**AFISCOOTER C**

require.


- ✓ Proceed with extreme caution when approaching the slope of a ramp or incline.
- ✓ Take your **AFISCOOTER C** wide around tight bends. When you do this, the rear wheels of the scooter will make a wide arc. Do not cut the bend and make sure that you do not hit or get caught on the corners of the railings.
- ✓ When descending a ramp, leave the speed of the **AFISCOOTER C** in the slowest setting to ensure a safe, controlled descent.
- ✓ Avoid abrupt stops and starts.
- ✓ Figure A helps to determine the maximum gradient as a function of the speed of the electric scooter.
- ✓ **AFISCOOTER C** speed 12 km/h: (450W)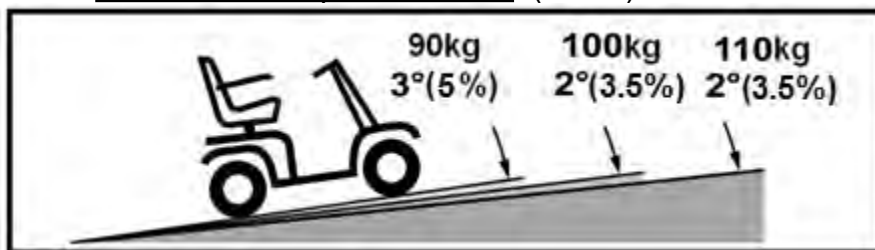


- ✓ **AFISCOOTER C** speed 10 km/h: (450W)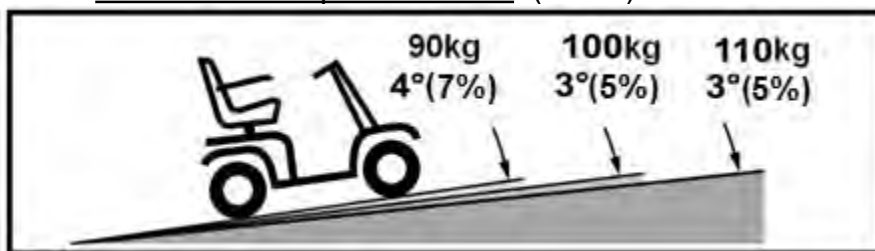


✓ **AFISCOOTER C** speed 15 km/h: (950W)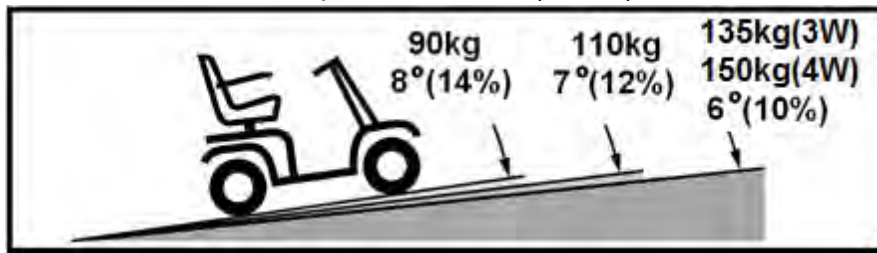

✓ **AFISCOOTER C** speed 12 km/h: (950W)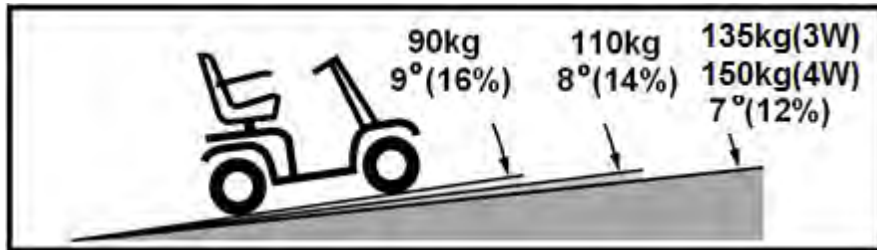

✓ **AFISCOOTER C** speed 10 km/h: (950W)

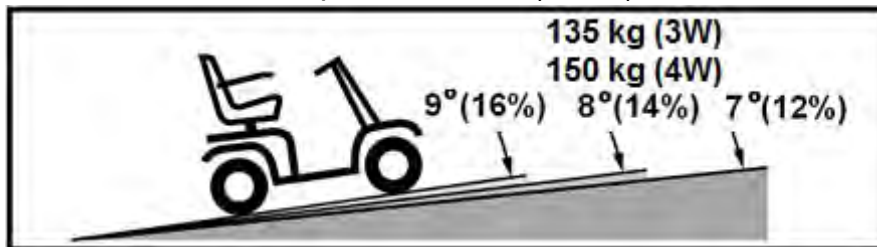**Fig. A: Recommended maximum angle of inclination**

When riding uphill, try to keep your **AFISCOOTER C** moving. If you have to stop, start up again slowly and then accelerate carefully. If you are driving down a slope, set the speed switch to MIN and only drive forwards. If your **AFISCOOTER C** starts to descend the slope faster than desired, let it come to a standstill by releasing the drive lever and then push the drive lever slightly forwards to ensure a safe and controlled descent.


**WARNING!** If you are sitting on or standing next to the **AFISCOOTER C** on a downhill slope, do not switch it to freewheel mode under any circumstances.

**WARNING!** When driving on a downhill slope, do not drive in a zigzag or diagonally upwards.

Always ride your **AFISCOOTER C** straight up the slope. This greatly reduces the risk of tipping over or falling. Always be extremely careful when riding up a slope.

**WARNING!** Do not drive your **AFISCOOTER C** diagonally up or down a slope; if possible, do not stop while driving up or down a slope.

**WARNING!** Do not drive up or down any potentially dangerous slopes (e.g. on surfaces covered with snow, ice, cut grass or wet leaves).

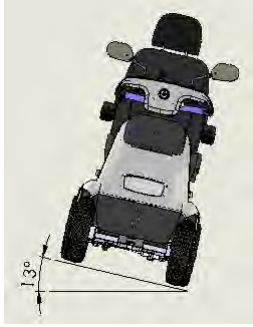**WARNING!**

Even if your **AFISCOOTER C** is capable of negotiating gradients greater than those shown in Figure A, you must under no circumstances exceed the specifications for riding on slopes or other technical data contained in this manual. Otherwise, your **AFISCOOTER C** could become unstable.

Please note that ramps for public access are not subject to government regulation in all countries and therefore do not necessarily have the same standard gradient. Other gradients may be naturally occurring or not specifically designed for electric scooters. Figure A illustrates the stability of your **AFISCOOTER C** and its climbing ability even under different weight loads and under controlled test conditions.

These tests were carried out with the **AFISCOOTER C seat** in the highest position and in the position furthest back. Use this information as a guideline. The ability of the **AFISCOOTER C** to ascend slopes is influenced by your weight, its speed, the angle of approach on the slope and the setting of the **AFISCOOTER C**.

**WARNING!** Any attempt to ride up or down a steeper slope than shown in Figure A may cause your **AFISCOOTER C** to become unstable and tip over.

**WARNING!** Never carry an oxygen cylinder weighing more than 6.8 kg. Never place more than 5 kg in the front basket or more than 8 kg in the rear basket.

**Figure B: Normal driving position Figure C: Driving position for increased stability**

When approaching an incline, it is best to lean forward. **See illustrations B and C.** This shifts the center of gravity of you and the **AFISCOOTER C** towards the front of the **AFISCOOTER C** for improved stability.

**NOTE: If the drive lever is released while driving forwards up a ramp, the AFISCOOTER C may roll back approx. 30 cm,**

***before the brake engages. If the drive lever is released while you are reversing, the AFISCOOTER C may roll back about one meter before the brake engages.***

### **INFORMATION ON CORNERING**

Excessive cornering speeds can lead to tipping. Factors that affect the risk of tipping include the speed at which you corner, the steering angle (how sharply you turn), uneven road surfaces, sloping road surfaces, driving from an area with low traction to an area with high traction (e.g. driving from a grass surface to a paved surface, especially at high speed while turning) and abrupt changes in direction. High speeds when negotiating bends are not recommended. If you have the feeling that you might tip over in a bend, reduce your speed and steering angle (i.e. reduce the sharpness of the bend) to prevent your **AFISCOOTER C** from tipping over.

**WARNING!** Reduce your speed when negotiating tight bends and keep your center of gravity stable. When riding the **AFISCOOTER C** at higher speeds, do not enter tight bends. This greatly reduces the risk of tipping over or falling. Always use common sense when cornering.

### **INFORMATION ON BRAKING**

Your **AFISCOOTER C** is equipped with two powerful braking systems:

- ✓ Regenerative braking system: Uses electricity to brake the vehicle quickly when the drive lever returns to the center/stop position.
- ✓ Parking brake: Activated automatically after the vehicle has been braked to a near standstill by regenerative braking or if the power supply to the system is interrupted for any reason.

### **OUTDOOR LANES**

- ✓ Your **AFISCOOTER C** is designed to provide optimum stability under normal riding conditions on dry, flat concrete or asphalt surfaces. However, we are aware that you will occasionally encounter other surfaces. For this reason, your **AFISCOOTER C** is designed to perform well on compacted soil, grass and gravel. You can use your **AFISCOOTER C** safely on lawns and in parks.
- ✓ Reduce the speed of the **AFISCOOTER C** if you are riding on uneven terrain and/or soft ground.
- ✓ Avoid tall grass, which can become entangled in the chassis.
- ✓ Avoid loose gravel and sand.
- ✓ If you feel unsafe on a riding surface, avoid it.

### **PUBLIC ROADS AND PATHS**

**WARNING!** Do not ride your **AFISCOOTER C** on public roads or paths.

It may be difficult for other road users to see you when you are sitting on your **AFISCOOTER C**. Observe all local regulations for pedestrian traffic. Wait until the path is clear and then proceed with extreme caution.

## FIXED OBSTACLES (STEPS, KERBS, ETC.)

**WARNING!** Do not drive near raised surfaces, unprotected projections and/or landings (kerbs, porches, stairs, etc.).

**WARNING!** Do not attempt to drive up or down an obstacle that is too high. Do not attempt to reverse down a step, kerb or other obstacle. This can cause

**AFISCOOTER C to tip over. WARNING! Make sure that your AFISCOOTER C is perpendicular to any kerb that you may have to negotiate.**

**WARNING!** Do not attempt to drive over a kerb that is higher than 5 cm.

## PRECAUTIONARY MEASURES IN BAD WEATHER

Avoid exposing your **AFISCOOTER C** to harsh weather conditions. If you are suddenly caught out by rain, snow, extreme cold or heat while riding your **AFISCOOTER C**, **move to** a sheltered area as quickly as possible. Dry your **AFISCOOTER C** thoroughly before storing, charging or operating it.


**FORBIDDEN!** Operation in rain, snow, salt, fog or spray mist and on icy, slippery surfaces can impair the electrical system.

**WARNING!** Prolonged exposure to extremely hot or cold weather conditions can affect the temperature of parts of the **AFISCOOTER C** and possibly cause skin irritation. Be careful when using your **AFISCOOTER C** in extremely hot or cold weather or if you expose your **AFISCOOTER C to** direct sunlight for a long period of time.

**WARNING** The surface temperatures can increase if they are exposed to external heat sources (e.g. sunlight).

## FREE RUN MODE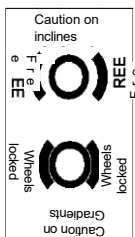

Your **AFISCOOTER C** is equipped with a manual freewheel lever which, when pressed forward, allows the **AFISCOOTER C** to be pushed. For more information on how to activate or deactivate the freewheel mode on the **AFISCOOTER C**, see chapter 4 "General description of the **AFISCOOTER C**".

**WARNING!** If your **AFISCOOTER C** is in freewheel mode, the parking system is disengaged.

- ✓ Only uncouple the drive motors on a level surface.
- ✓ Make sure that the key is removed from the key switch.
- ✓ Stand next to the **AFISCOOTER C** to activate or deactivate freewheel mode. Never do this while you are sitting on the **AFISCOOTER C**.
- ✓ After you have finished pushing the **AFISCOOTER C**, always return to drive mode to engage the brakes (pull the manual freewheel lever backwards).
- ✓ An additional function built into the **AFISCOOTER C** is **push-too-fast** protection, which protects it from excessive speed in freewheel mode.

**Push-too-fast** works in two ways:

- ✓ If the key is in the OFF position while the **AFISCOOTER C** is in freewheel mode, the **AFISCOOTER C control** unit activates regenerative braking if it is pushed faster than the pre-programmed upper speed limit allows. In this case, the control unit acts as a speed regulator.
- ✓ If the key is in the ON position while the **AFISCOOTER C** is freewheeling, you will encounter considerable resistance at any speed. This prevents the **AFISCOOTER C** **from** unintentionally gaining momentum if the freewheel lever is accidentally actuated while riding the **AFISCOOTER C**.
- ✓ When using this lever, ensure that you do not touch the inside surface of the motor, as it can be very hot and cause injury. Observe the warning sign on the back of the scooter near the lever.

## STAIRS AND ESCALATORS

**AFISCOOTER C** electric scooters are not suitable for use on stairs or escalators.

Always use an elevator.

**WARNING!** Do not use your **AFISCOOTER C** to climb stairs or escalators.

## DOORS

- ✓ Determine whether the door opens towards you or away from you.
- ✓ Use your hand to turn the door knob or to push the door handle or push bar.
- ✓ If the door opens away from you, drive your **AFISCOOTER C** carefully and slowly forwards to push the door open.  
If the door opens towards you, reverse your **AFISCOOTER C** carefully and slowly to pull the door open.

## TRAINING

Modern elevators have a safety mechanism at the edge of the door that opens the door(s) again when it is operated.

- ✓ If you are in the doorway of an elevator and the door(s) start to close, press on the rubber edge of the door or allow the rubber edge of the door to touch the **AFISCOOTER C**; and the door will open again.
- ✓ Make sure that handbags, parcels or **AFISCOOTER C** accessories cannot get caught in the elevator doors.

**NOTE:** Sometimes maneuvering with the **AFISCOOTER C** in elevators and building entrances can be difficult. Be careful when maneuvering with your **AFISCOOTER C** in confined spaces and avoid areas that could pose a problem.

## LIFTS AND HOISTS

When traveling with your **AFISCOOTER C**, it may be necessary to use a lift or lifting device to facilitate transport. We recommend that you read the manufacturer's instructions, specifications and safety information carefully before using such a lifting device.

**WARNING!** Never remain seated on the **AFISCOOTER C** when it is on a lifting device.

Your **AFISCOOTER C** was not designed for such use and you are solely responsible for any damage or injury resulting from such use.

## **BATTERIES (see also "Battery replacement", page 28)**

In addition to the warnings listed below, observe all other information on handling the battery.

**FORBIDDEN!** Battery terminals, clamps and corresponding accessories contain lead and lead compounds. Wear safety goggles and gloves when handling batteries and wash your hands after handling.

**WARNING!** The batteries of the scooter are heavy. (see Appendix A "Technical data"). If you are unable to lift this much weight, you should always seek assistance. Use suitable lifting techniques and avoid lifting beyond your capacity.

**WARNING:** Always protect the batteries from freezing and never charge a frozen battery.

**WARNING:** Connect the battery cables correctly. The RED cables (+) must be connected to the positive terminals or poles (+) and the BLACK cables (-) must be connected to the negative terminals or poles (-) of the battery. Replace damaged cables immediately. Protective caps must be fitted over all battery terminals.

**NOTE:** *If the battery is damaged or cracked, immediately place it in a plastic bag and contact your local waste disposal facility or an authorized **AFISCOOTER C** dealer for disposal instructions.*

## **TRANSPORTATION ON A MOTOR VEHICLE**

The manufacturer does not recommend that you remain seated on the **AFISCOOTER C** during transportation on a motor vehicle. The **AFISCOOTER C** should be stowed in the trunk of a car or on the loading area of a truck or van with the batteries removed and properly secured. In addition, all removable parts of the **AFISCOOTER C**, including the armrests, seat and cover, should be removed and/or properly secured during transportation.

**WARNING:** Although your **AFISCOOTER C** may be equipped with an optional restraint belt, this belt is not designed for restraint during transportation in a motor vehicle. Any person riding in a motor vehicle must be properly restrained in the motor vehicle seat with seat belts securely fastened.

**WARNING:** Do not sit on the **AFISCOOTER C** while this scooter is in a moving vehicle.

**WARNING!** Always make sure that the **AFISCOOTER C** and its batteries are properly secured when being transported. The batteries must be secured in an upright position and protective caps must be fitted to the battery terminals. Batteries must not be transported together with flammable or combustible objects.

## **HOLDING STRAP**

Although your **AFISCOOTER C** may be equipped with an optional restraint belt, this belt is not designed for restraint during transportation in a motor vehicle. Any person riding in a motor vehicle must be properly restrained in the motor vehicle seat with seat belts securely fastened.

**WARNING:** Do not sit on the **AFISCOOTER C** while this scooter is in a moving vehicle.

**WARNING:** Make sure that the **AFISCOOTER C** and its batteries are properly secured when it is being transported.

### PREVENT UNINTENTIONAL DRIVING

**WARNING:** Switch off the main switch of the scooter if it is not expected to be used for a long period of time. This prevents unexpected driving caused by accidentally touching the drive lever.

### GETTING ON AND OFF THE **AFISCOOTER C**

Getting on and off the **AFISCOOTER C** requires a good sense of balance. Please observe the following safety instructions when getting on and off the **AFISCOOTER C**:

- ✓ Remove the key from the key switch.
- ✓ Make sure that the **AFISCOOTER C** is not in freewheel mode
- ✓ (see chapter 4 "General description of the **AFISCOOTER C**").
- ✓ Make sure that the seat is properly secured.
- ✓ Swivel the armrests upwards.

**WARNING:** Position yourself as far back in the seat as possible to prevent the **AFISCOOTER C** from tipping over and causing injury.

**WARNING!** Avoid shifting your entire weight onto the armrests of the **AFISCOOTER C** and do not load the armrests with weights, e.g. during transportation. Such use can cause the **AFISCOOTER C** to tip over, which can result in a fall from the **AFISCOOTER C** and/or injuries.

**WARNING!** Avoid shifting your entire weight to the ground. This can cause the **AFISCOOTER C** to tip over.

### STRETCH AND BEND

Avoid stretching and bending while riding the **AFISCOOTER C**. If you lean forward, there is a risk of accidentally touching the control lever. If you lean to the side while seated, there is a risk of tipping over. It is important to maintain a stable center of gravity so that the **AFISCOOTER C** does not tip over. We recommend that you determine your personal limits and practise bending and stretching in the presence of a qualified assistant.

**WARNING!** Do not reach for objects by bending, stretching or leaning over them in order to pick them up from the **AFISCOOTER C** floor or from one side of the **AFISCOOTER C**. This can lead to a change in your center of gravity and the weight distribution of the **AFISCOOTER C**, which can cause it to tip over.

**FORBIDDEN!** Keep your hands away from the tires and wheels while driving. Note that loose-fitting clothing can get caught in the tires and wheels.

### PRESCRIPTION DRUGS AND PHYSICAL LIMITATIONS

The **AFISCOOTER C** user must exercise care and common sense when riding the **AFISCOOTER C**. This also includes awareness of

Safety issues when taking prescription or over-the-counter medications or if you have certain physical limitations. **WARNING:** Talk to your doctor if you are taking prescription or over-the-counter medication or if you have certain physical limitations. Some medications and limitations may affect your ability to operate the **AFISCOOTER C** safely.

### ALCOHOL AND SMOKING

The user of the **AFISCOOTER C** must exercise caution and common sense when operating the **AFISCOOTER C**. This also includes being aware of safety problems under the influence of alcohol or when smoking. **WARNING!** Do not ride the **AFISCOOTER C**, while under the influence of alcohol, as this may impair your ability to operate the **AFISCOOTER C** safely.

**WARNING!** The manufacturer strongly advises against smoking while sitting on the **AFISCOOTER C**, even though the **AFISCOOTER C seat** has passed the necessary test requirements for resistance to ignition in accordance with EN 1021-2:2006. You must adhere to the following safety guidelines if you decide to smoke cigarettes while sitting on your **AFISCOOTER C**.

- ✓ Do not leave burning cigarettes unattended.
- ✓ Keep ashtrays at a safe distance from the seat cushions.
- ✓ Always make sure that the cigarettes are completely extinguished before disposal.

### ELECTROMAGNETIC AND RADIO INTERFERENCE (EMI/RFI)

**WARNING!** Laboratory tests have shown that electromagnetic and radio frequency waves can affect the performance of scooters. Electromagnetic (EMI) and radio frequency interference (RFI) can be generated by sources such as cell phones, mobile two-way radios (e.g. walkie-talkies), radio transmitters, television transmitters, amateur radio (HAM) transmitters, wireless computer links, microwave signals, paging transmitters and medium-range mobile transceivers used by emergency vehicles. In some cases, they can cause unintentional movement or damage to the controls. Every electric vehicle is protected against electromagnetic interference to a certain degree.

The higher the immunity level, the greater the protection against electromagnetic interference. This product was tested with an immunity level of 20 V/m and passed this test.

**WARNING!** Please note that cell phones, two-way radios, laptops and other types of radio transmitters can cause unintentional movements of your scooter.

**AFISCOOTER C** due to electromagnetic interference. Be careful when using one of these devices when operating your **AFISCOOTER C** and avoid proximity to radio and television transmitters.

**WARNING!** Adding accessories or components to a scooter may increase its susceptibility to electromagnetic interference. Do not modify your **AFISCOOTER C** in any way that is not authorized by the manufacturer.

**WARNING!** The **AFISCOOTER C** itself can interfere with other electrical devices in the vicinity, such as alarm systems.

**NOTE:** Further information on electromagnetic and radio interference can be found in chapter 10 or can be obtained from your authorized dealer.

## CHAPTER 4 - GENERAL DESCRIPTION OF THE AFISCOOTER C

Note: The figures in brackets refer to the parts shown in Figures 1, 2 and 3.

Figure 1

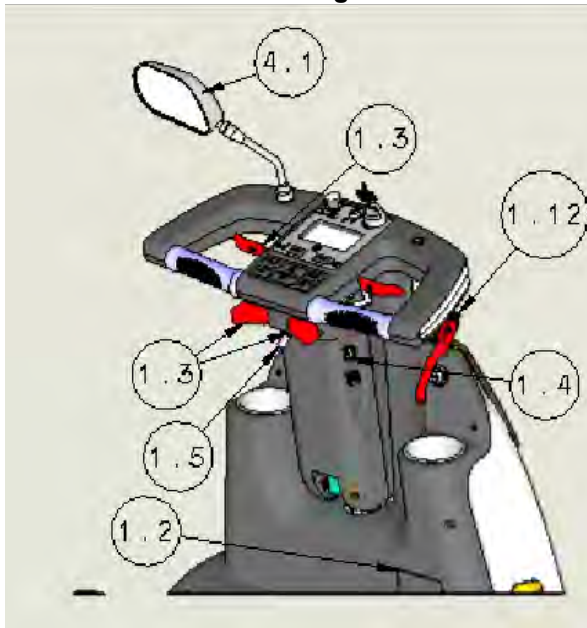
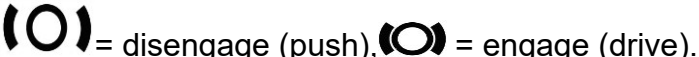
Figure 2

Figure 3

Label with serial number and manufacturer's data (1.2)	Serial number and manufacturer data can be found on a sign on the mud flap in front of your feet.
Steering column (1) / Handlebar (1)	The <b>AFISCOOTER C</b> is designed to be operated by both right-handed and left-handed users, but is supplied with right-handed controls. Controls for left-handed users can be set up by your specialist dealer.
Rear-view mirror (4.1)	Each <b>AFISCOOTER C</b> has either 1 or 2 rear-view mirrors.
Drive lever (1.3)	Used to control the speed of the <b>AFISCOOTER C</b> is used.
Brake lever (1.12)	Used for braking in emergencies or when required.
Charging socket (1.4)	Used to connect the charging plug for charging of the battery.
Handlebar adjustment lever (1.5)	The handlebars can be adjusted forwards and backwards. Release the lever, make the adjustment and move the lever to the desired position. back into the locked position.
Front basket (1.7)	For the transportation of parcels and personal Objects. Maximum load capacity 5 kg.
Headlights (1.8) and Front indicators (1.8)	(A) Headlights for poor visibility (B) Blinker for turn signals
USB charger (1.20)	Used for a cell phone charger.
Battery cover (2)	Protects the drive unit, the electronics and the Batteries.
Fuses (2.1a, 2.1b)	2.1a - Main fuse 70 A - semi-automatic 2.1b - 10 A charging fuse - semi-automatic If there is a fuse between the rear wheel and the Battery cover
Manual lever for releasing the electromagnetic brake (EMB) (2.2)	A manually operated release lever at the rear of the <b>AFISCOOTER C</b> can be pushed or pulled. Press or pull the lever as indicated on the label:  (○) = disengage (push), (◓) = engage (drive). If the lever is in the disengaged position, the <b>AFISCOOTER C</b> cannot be operated or driven. <b>Note:</b> After the lever has been operated, you must reset the system, i.e. switch the key switch to the OFF position and back to the ON position. <b>Note:</b> Never move the lever to the release position when the <b>AFISCOOTER C</b> is on an incline.
Rear bumper (2.2)	
Tail lights and stop lights (2.2)	
Rear indicators (2.2)	

Seat (3)	The <b>AFISCOOTER C seat is</b> equipped with folding armrests (7.3) to allow easy access to the seat. The seat itself can be swiveled for easy access.
Armrests (3.4)	Are used for comfortable entry. Can be folded upwards.
Seat adjustment lever (3.1)	Is used to adjust the seat forwards and backwards. used at the back.
Seat swivel lever (3.2)	Used to swivel the seat by 45° or 90° in both directions.
Floor (4)	Covered with carpet, which is removed for cleaning can be used.

## CHAPTER 5 - SETTING INSTRUCTIONS

(The figures refer to Figures 4 and 5.)

Steering column angle adjustment (1.5)	<p>The steering column angle is set as follows:</p> <ul style="list-style-type: none"> <li>• Release you the steering column adjustment lever (1.5) by pressing it down.</li> <li>• Release the lever at the desired position.</li> </ul>
Adjusting the handlebar height	<p>The handlebar height can be adjusted by a technician by approx. 25 mm.</p>
Sitting on the <b>AFISCOOTER C</b>	<p>Before riding the <b>AFISCOOTER C</b>, <b>make</b> sure that the plug is disconnected from the charging socket (1.4).</p> <p><b>Note:</b> If the plug is in the charging socket, the <b>AFISCOOTER C</b> cannot be operated.</p> <p>Use the seat swivel lever (3.2) to turn the seat to the side and lift the armrest (3.4) so that you can sit comfortably on the seat.</p> <p>You can use the seat adjustment lever (3.1) to adjust the position of the seat forwards and backwards.</p>
Handlebars and Hand protectors	<p>Always keep your hands on the handlebars when riding (1) and inside the hand protectors.</p>
Adaptation of the shock absorbers to the user	<p>It is possible to adjust the rear shock absorbers to the desired load or type of use of the AFISCOOTER C. If necessary, ask a technicians to adjust the shock absorbers to your needs.</p>

Figure 4

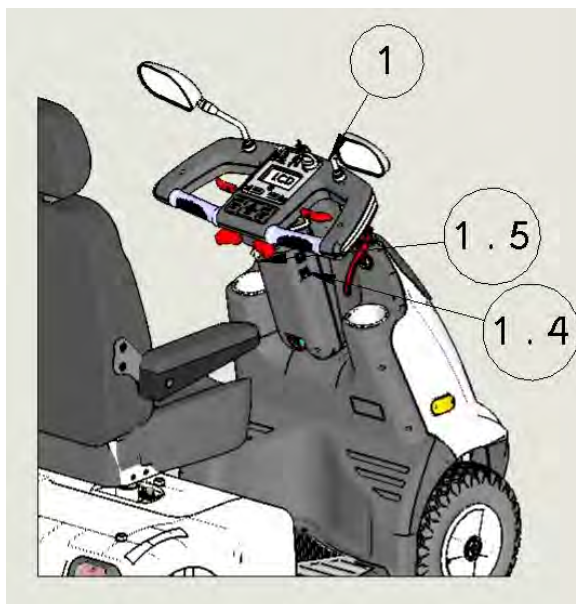
Figure 5

# CHAPTER 6 - OPERATING INSTRUCTIONS

## 6.1 GENERAL

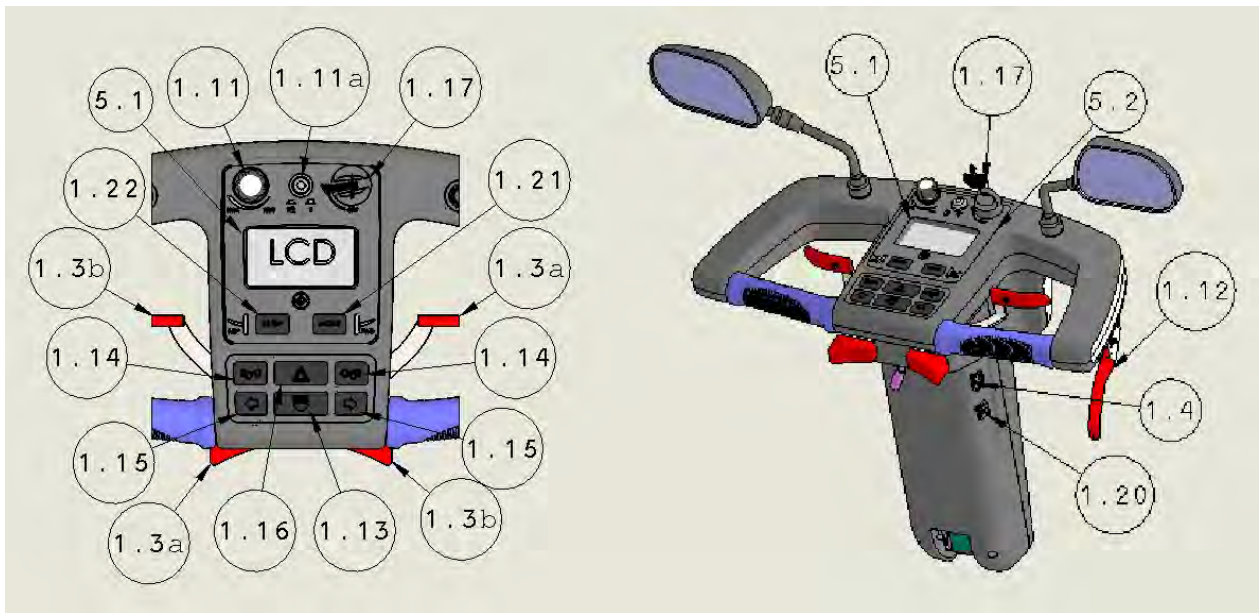
Figure 6

(The numbers refer to figures 6 and 7.)


<p>Key switch (1.17)</p>	<ul style="list-style-type: none"> <li>The master key switch is located at the top of the steering column (1).</li> </ul> <p>The key switch has two positions:</p> <ul style="list-style-type: none"> <li>✓ <b>OFF</b> - The <b>AFISCOOTER C</b> cannot be driven and the electromagnetic brake (EMB) is applied.</li> <li>✓ <b>ON</b> - The <b>AFISCOOTER C</b> is supplied with power and the electromagnetic brake (EMB) is automatically released as soon as the drive lever is pressed.</li> </ul> <p>To drive the <b>AFISCOOTER C</b>, insert the key into the key switch and turn it to ON (clockwise). The switch remains in this position and the green LED (5.2) lights up.</p> <p><b>Note:</b> The key switch is used to operate the <b>AFISCOOTER C</b>, not for locking.</p>
<p>Automatic switch-off</p>	<p>If the key switch is set to ON, the control system is automatically switched off after a few minutes (normally 20) to conserve the battery.</p> <p>For automatic switch-off, turn the key switch to OFF and then back to ON.</p>
<p>Setting the handbrake</p>	<p>The handbrake cable mechanism can be adjusted by an authorized technician.</p> <p><b>The handbrake is part of the safety system.</b></p> <p><b>Do NOT set them yourself, but get professional help!</b></p>
<p>Driving lever and handbrake</p>	<p>Driving: Press the drive lever (1.3) lightly until you have reached the desired speed. Your speed is directly proportional to the deflection of the drive lever.</p> <p>Stopping: Release the drive lever (1.3). The <b>AFISCOOTER C</b> stops automatically and gradually.</p> <p>Manual parking brake - push the lever as far as it will go and press the button on the side of the lever.</p>

	<b>Note: You can also use the manual parking brake (1.12) for emergency braking.</b>
Signaling	<p>Before turning, press the corresponding blinker switch left or right (1.15). To switch off the blinker, press the switch forwards.</p> <p>If a blinker is activated, the corresponding LED flashes. Depending on the model, a signal tone may also sound (depending on the local regulations). To switch off the display, press the button again.</p>
Signal tone buttons (1.14)	<p>Press one of the 2 buttons (1.14) to activate the signal tone. The signal tone sounds automatically under the following conditions (<i>unless this is prohibited by law</i>):</p> <ol style="list-style-type: none"> <li>1. When reversing</li> <li>2. When operating the left or right turn signal <u>Note:</u> It is possible to change the audible signal function by changing a DIP switch on the control panel. which must be carried out by an authorized technician.</li> </ol>
Headlight switch (1.13)	<p>To switch on the headlights and tail lights, press the lighting button (1.13).</p> <p>When the lighting is switched on, the lighting LED (5.6) lights up. To switch off the lighting, press the button again.</p>
Flashing warning button (1.16)	<p>The hazard warning lights button activates all the flashing lights on the <b>AFISCOOTER C</b>. To switch off the hazard warning lights, press the button again.</p> <p>When the hazard warning lights are activated, both LEDs (5.3 and 5.4) flash.</p>
Locking during loading	<p>When the AFISCOOTER C is loaded, it is in the Lock mode. It cannot be driven in this mode. A signal lamp lights up during the charging process.</p>
Switch for safe driving at half speed (1.11)	<p>Drive very slowly and carefully when driving on ramps, high kerbs and sharp bends. You should set the switch for safe driving (1.11) to half speed mode. Never attempt to drive over an obstacle that exceeds the maximum recommended angle of inclination and the obstacle height specified in the data sheet.</p> <p><b>Always drive carefully and adapt your speed to the road conditions.</b></p> <p><b>Reduce the speed of the vehicle before sharp bends.</b></p> <p><b>AFISCOOTER C to half speed to prevent it from tipping over.</b></p>
LCD (5.1)	<p>When driving and operating the AFISCOOTER C, all necessary information is shown on the LCD display</p>
MODE and RESET buttons (1.21)	<p>The MODE/RESET button switches the digital display from speed to distance or time. The distance traveled is displayed in km or miles and the time in hours.</p> <p><b>(Note: the speed unit can be set by a technician).</b></p>

	When you press the MODE button to select the "Travel time" and Press "Travel distance" and hold for longer than 7 seconds to reset the travel time and distance to zero.
Setup button (1.22)	The setup button is used to set the clock.
Parking	After stopping the <b>AFISCOOTER C</b> , <i>turn</i> the key switch (1.17) to OFF and remove the key. The LED switches off. <b>Note: The electromagnetic brake (EMB) is automatically locked when the drive lever is released and the AFISCOOTER C has come to a complete standstill.</b>

## 6.2 USE OF THE LC DISPLAY

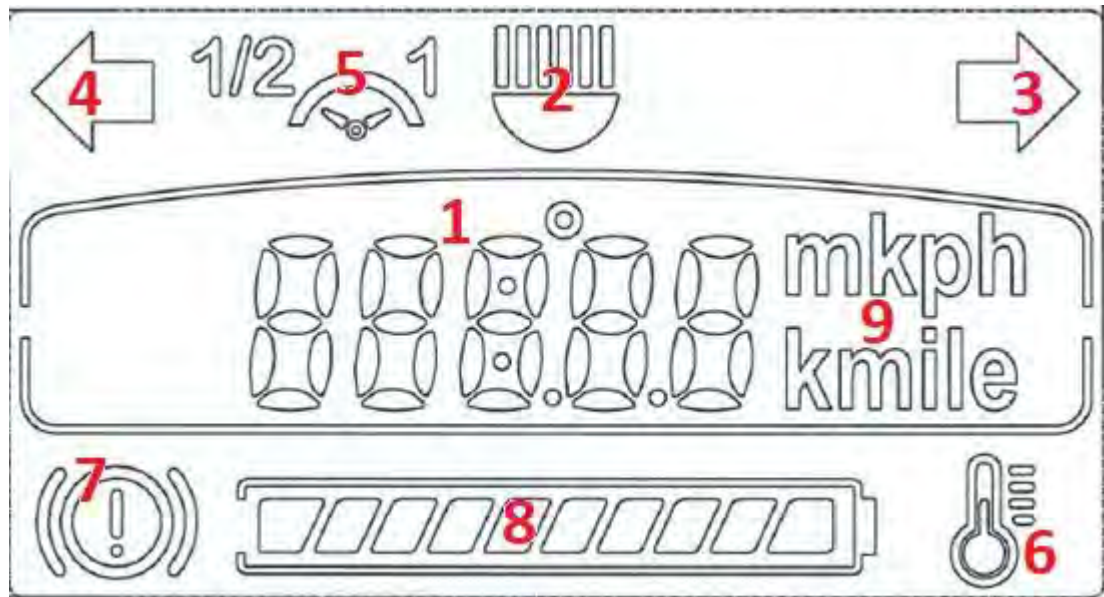LC display button	When driving and operating the AFISCOOTER C, all necessary information is shown on the LC display (5.1). See details below.
Battery status display	The <b>battery capacity indicator</b> (8) on the LC display shows the remaining battery capacity. How the The fuel gauge in the car is not absolutely accurate; it is merely a guide to prevent you from running out of energy. The accuracy of the gauge also depends on the type and age of the batteries. If the gauge shows 2 to 3 bars, charge the batteries as soon as possible. If the display shows a flashing bar, recharge immediately. You can only ride a short distance (3 to 4 km on level ground). The accuracy of the battery display also depends on the type and age of the batteries.
Fault display	The <b>AFISCOOTER C</b> is equipped with a fault indicator. The fault display appears as a flashing code on the battery display and as an error code on the LC display (1.1). Further details can be found in the maintenance instructions for the AFISCOOTER.
Lighting display	When the main lighting is switched on (front and back), a symbol lights up (2).
Turn signal indicators	When the indicators are switched on, the symbols 3 and/or 4.
Display for half speed	When the switch for half speed (1/2) is in the half speed position, the "1/2" symbol lights up (5).

Alarm display	If one of the following conditions exists, the alarm indicator (7) lights up: Engine is overheated, the panic brake has been applied or the manual release lever of the electromagnetic brake (EMB) is in the OPEN position.	
Speed display (1)	The LC display shows the speed of the <b>AFISCOOTER C</b> digitally. It is displayed in kph (km/h) or mph (miles per hour) is displayed. A technician can set the unit for the speed.	
Indication of high motor temperature (6)	Not relevant	
Setting the clock	If you switch to time mode with the MODE button and hold it down for 7 seconds, you will enter time setting mode. <b>Setting the clock:</b> Use the left button <b>Setup</b> and the right button <b>Fashion</b> as follows: Each press of the <b>Mode</b> button changes the time unit and the selected unit flashes. During flashing, each press of the <b>Setup</b> button increases the unit by 1.	
	<b>Level</b>	<b>Display</b>
	1	HH:mm
	2	HH:mm
	3	DD.MM or MM.DD
	4	DD.MM or MM.DD
	5	20YY
6		Saving the updated Time and date

## 6.3 DRIVING AND GENERAL USE

Overcoming obstacles	To overcome an obstacle, drive the <b>AFISCOOTER C</b> forwards towards the obstacle until the front wheel touches the obstacle and then increase your speed while maintaining your speed in the direction of travel.
Gradients and curves	<p>Reduce your speed to a minimum before sharp bends, especially on inclines.</p>  <p>minimum to prevent tipping over.</p> <p>The <b>AFISCOOTER C is capable</b> of climbing slopes with a greater angle of inclination than the recommended safe angle of inclination. Do not ride on inclines where you are not sure of the degree of incline. However, drive very slowly and avoid sideways angles of more than 18% (10°).</p>
Before use	After reading through this entire manual and before using the <b>AFISCOOTER C</b> , carry out a visual inspection of the vehicle to ensure that there is no visible damage. If you have any questions or concerns, please contact Please contact your dealer.
Programming the control unit	Several parameters of the <b>AFISCOOTER C</b> control <b>unit</b> can be programmed. Programming may only be carried out by a trained, authorized technician. Incorrect programming can cause abnormal operation of the <b>AFISCOOTER C</b> and cause damage and injury.
Function test	<p>A function test is strongly recommended when you receive your <b>AFISCOOTER C for the</b> first time or after a period in which you have not used the <b>AFISCOOTER C</b>. Proceed as follows:</p> <ol style="list-style-type: none"> <li>1. Switch on the key switch.</li> <li>2. Make sure that both the green ON LED and the battery status indicator light up.</li> <li>3. Slowly press the drive lever and check whether the <b>AFISCOOTER C</b> moves at a speed that corresponds to the degree to which you press the lever.</li> <li>4. Release the drive lever and make sure that the <b>AFISCOOTER C</b> stops gently and that you hear the click of the electromagnetic brake (EMB) engaging.</li> <li>5. Check that all buttons on the handlebars are working properly.</li> <li>6. Turn off the key switch. Your <b>AFISCOOTER C</b> is now ready to drive.</li> </ol>
Before the Use	Before using the AFISCOOTER C, make sure that your own weight and the weight of objects,

	<p>that you are transporting on the device. The maximum payload is 135 kg for models with 3 WHEELS or 150 kg for models with 4 WHEELS (depending on model).</p>
<p>Security /seat belt</p>	<p>Always wear the seat belt when it is required as a front pelvic support or when required by local traffic regulations. A seat belt can be ordered from your local dealer (see optional equipment for <b>AFISCOOTER C</b> in Appendix A).</p>
<p>Save the <b>AFISCOOTER C</b> in a motor vehicle for transportation</p>	<p>If the <b>AFISCOOTER C</b> is to be transported in a motor vehicle, it must be secured (see Figure 8). Attach the 4 yellow painted eyelet plates (2 on the front and 2 on the rear of the scooter) to the car's securing hooks using commercially available, approved tensioning straps. <b>Warning: No one may sit on the AFISCOOTER C while it is being transported.</b></p> <div data-bbox="609 712 1297 1451" data-label="Image"> <p>The diagram shows a side view of the AFISCOOTER C scooter. It is a three-wheeled vehicle with a seat and handlebars. Four yellow eyelet plates are attached to the scooter: two on the front and two on the rear. Two standard belts and a towing hook are attached to the front eyelets, and two standard straps are attached to the rear eyelets. The text 'STANDARD STRAPS + CAR HOOKS' is written below the diagram. Labels with arrows point to 'Eyelets at the front Scooter', 'Eyelets at the back of the scooter', and 'standard belts and towing hook + CAR HOOKS'.</p> </div> <p style="text-align: center;"><b>Figure 8</b></p>
<p>Transportation</p>	<p>To reduce the height of the <b>AFISCOOTER C</b> during transportation, the seat can be removed as described in chapter 8 "Maintenance" and the steering column folded into the horizontal position. become.</p>

## CHAPTER 7 - LOADING

Read the instructions for the charger before using it.

**Warning: If you use a charger other than the one supplied with your *AFISCOOTER C* supplied, ask your dealer for instructions.**

### **CHARGING THE BATTERIES** (see Figure 9)

1. Insert the charging plug (8) into the charging socket (10) located on the right-hand side of the steering column.
2. Plug the mains cable of the charger (9) into a wall socket (11).
3. When the charging process is complete, unplug the mains cable from the charger (9) from the socket (11) and then remove the charger plug (8) from the charging socket (10).
4. Under ideal storage conditions, batteries that have been fully charged and not used should be recharged every 6 months.  
If you do not expect to use your ***AFISCOOTER C*** for a longer period of time, we recommend charging it for two days and then disconnecting the batteries.
5. If you have not used your ***AFISCOOTER C*** for a long time, charge the batteries for at least 24 hours before riding.

### **Recommended loading routine**

1. Use the scooter throughout the day as required or until the charge level is low, according to the battery indicator on the LC display.
  2. Charge the batteries of the electric vehicle overnight at the end of the day.
  3. The batteries do not need to be disconnected at the end of the charging process. The charger works automatically and switches itself off.
  4. It is not necessary to charge the scooter's batteries at the end of each journey, unless you do not intend to use the scooter that day.
  5. If you charge the electric scooter during the course of the day, you should carry out a full charge.
  6. It is not recommended to leave the batteries uncharged for several days, as this can reduce the driving distance and the service life of the batteries.
- **As long as the charging plug is inserted in the charging socket, the electronic control unit of the *AFISCOOTER C* automatically interrupts the power supply to the electrical system and the vehicle cannot be driven.**
  - **The charger supplied is suitable for charging lead-acid batteries (dry or gel batteries). Only use the specified charger type. Ask your dealer before using a different charger.**

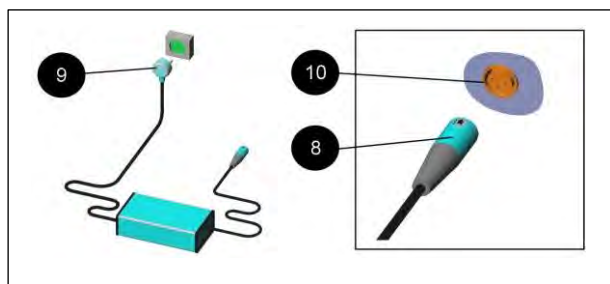
Figure 10

## CHAPTER 8 - MAINTENANCE

### Note:

1. Maintenance instructions are available for trained technicians.
2. The figures in brackets refer to the parts shown in Figure 11.

Although the **AFISCOOTER C** requires very little maintenance, you should not neglect it.

The expected service life of the **AFISCOOTER C** is 7 years and can be considerably longer with careful care and maintenance.

When carrying out maintenance work on the **AFISCOOTER C**, make sure that it is standing on a level surface and that the key is not inserted.

### Regular maintenance work

✓ **Note:** No special tools are required for regular maintenance.

No.	Type of work	Frequency
1	Check the air pressure in all tires	Every week
2	Check emergency brakes	Every week
3	Normal driving and Check stopping	Every month
4	Check tire wear	Every month
5	Check for missing or damaged parts (see Figures 1 to 4)	After receiving the <b>AFISCOOTER C</b> or after a longer period without use
6	Fixed seat of all screws and parts check	Once a year through a Technician
7	Battery replacement <b>(To replace the batteries, the seat and the battery cover (located under the seat) must be removed).</b>	<ul style="list-style-type: none"> <li>• Every 2 to 3 years</li> <li>• After around 300 to 400 complete charging /discharge cycles</li> <li>• If the distance you can cover on a single charge becomes short or</li> <li>• After recommendation by a Technician</li> </ul>
  <b>Used batteries must not be disposed of with household waste. Information on the proper disposal of batteries can be obtained from your authorized specialist dealer.</b>		
8	Cleaning	Outside - if required Inside - once a year by a technician

### Recommended periodic functional tests

#### Slow or sluggish movement

If the scooter does not run at full speed and the battery is in good condition, check the position of the speed limiter. If adjusting the speed limiter does not solve the problem, there may be a harmless fault.

Contact your workshop.

### Daily checks by the user

Drive lever: With the scooter switched off, check that the drive lever mechanism is not bent or damaged and that it returns to its position when pressed and released. If a problem occurs, do not continue with the safety checks and contact your workshop.

### Weekly checks by the user

Drive lever: Move the drive lever to the "Full speed forward" position and switch the scooter on. The scooter should not move. The display shows that you have switched on the scooter with the drive lever already depressed.

If the scooter moves, contact your workshop. Parking brake: This test should be carried out on a level surface with at least one meter of free space around the scooter.

Switch on the scooter.

Check whether the status display is still lit or flashing slowly after half a second.

Drive the scooter slowly in a forward direction until you hear the parking brake engage.

The scooter may start to move. Release the drive lever immediately. You should be able to hear the parking brake activate within a few seconds.

Repeat the test in the opposite direction. Cable and plug:

Check that all plugs on the scooter are securely plugged in and ensure that all cables are free from damage.


## **Detailed procedures for demand maintenance**

<b>Type of work</b>	<b>Frequency</b>
Remove seat and battery cover (2)	Turn the seat 45° (in one of the two directions) by turning the lever (7.5). The seat can only be removed at this angle. Removing the seat -(7) Hold the seat with one hand on the backrest and with the other on the front and lift it off the seat column.
	<b>Warning! The seat weighs 20 kg.</b> The seat must be lifted by two people or with a suitable lifting device. Refit the battery cover, place it over the seat pillar, make sure it is correctly positioned and secure it. Then secure it with the five screws.
Remove and refit the battery cover (2)	To refit the battery cover, place it back over the seat pillar and make sure it is properly seated, then secure it with the four screws (2.6).
Batteries	Each battery weighs 23 kg. The batteries must be lifted by two people or with a suitable lifting device. become.
Tires	The correct tire pressure is crucial for steering and stability. Check the tire pressure every two weeks. 25 ±2 psi Front tire on the AFISCOOTER version with 4 Wheels: 20 ±2 psi


Type of work	Frequency
Cleaning	Only use a damp cloth and a mild cleaning agent for cleaning. Never use a hose, as this can cause serious damage to the drive and motor components. electronic components.
Battery replacement	<p><b>Please note!</b> Batteries may only be replaced by trained specialist personnel.</p> <p><b>Warning!</b> Batteries contain a lot of energy and can cause sparks, which can lead to a potential fire hazard.</p> <p><b>Warning!</b> Improper handling of batteries can lead to damage and injury.</p> <p>Please refer to Figure 12 for this procedure.</p> <ol style="list-style-type: none"> <li>1. Remove the terminal insulation cap from a battery terminal.</li> <li>2. Unscrew the cable from the terminal using a 10/13 mm wrench. Make sure that the wrench does not touch any other terminals or metal parts, as this can lead to damage or injury.</li> <li>3. Repeat steps 1 and 2 for all other terminals</li> <li>4. Remove the old batteries and insert the new batteries.</li> <li>5. Connect the cables of the new batteries as shown in Figure 11 (see also the label near the batteries).</li> <li>6. Screw the clamps tight and insert the Replace the insulating caps.</li> </ol>

**Shield for battery wiring**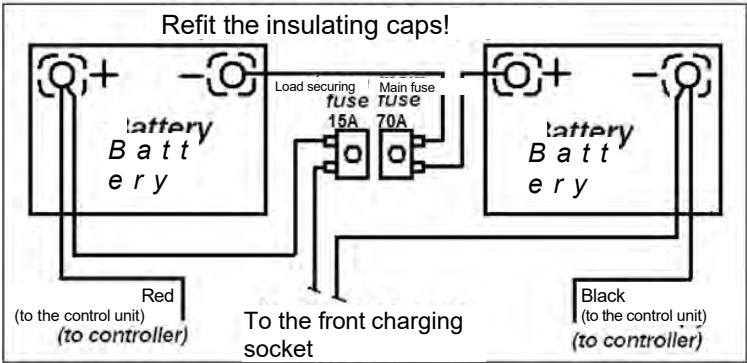

**Instructions for wiring and rear mounting**

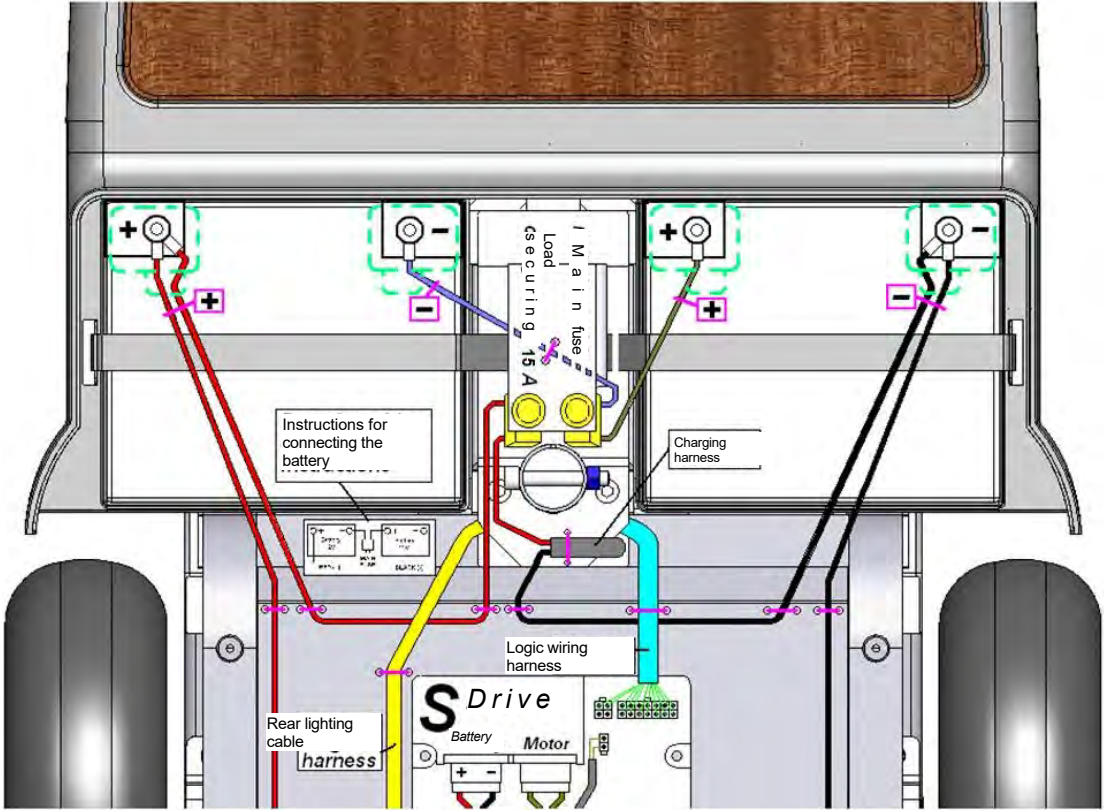**Figure 11**

## CHAPTER 9 - STORAGE

---

For long-term storage, e.g. longer than four months, it is advisable to store the **Afiscooter C** *in* a dry, clean and safe place, ideally with access to a power socket so that you can charge it.

During storage, the **Afiscooter C** should be covered and the main switch should always be switched off. If the scooter is not used regularly, charge the battery at least once a month until normal use is resumed. If you leave the main switch switched on or do not charge the scooter for a longer period of time, the batteries could discharge. We recommend disconnecting the terminals from the battery poles.

It is not recommended to store the **Afiscooter C** in extremely cold conditions. Storage at temperatures below 10° Celsius will reduce the runtime and battery capacity.

**WARNING: DO NOT ALLOW THE VEHICLE TO FREEZE, DO NOT STORE THE VEHICLE AT TEMPERATURES BELOW 0 °C**, this could permanently damage the battery.

## CHAPTER 10 - TROUBLESHOOTING AND REPAIR

The following table contains instructions for troubleshooting and repairing problems that may occur when operating the **AFISCOOTER C**, as well as the best method for solving the problem. The table contains three columns:

Symptom	Probable cause	Remedy
The problem you have	The most likely causes, in order of probability (most most likely first)	Recommended actions to address each probable cause

**Remember:** you can always contact your dealer for further professional support.

No.	Symptom	Probable causes	Remedy
1	Flat tires and unstable handling	<ul style="list-style-type: none"> <li>✓ Low tire pressure</li> <li>✓ Hole in the tire</li> </ul>	<ul style="list-style-type: none"> <li>✓ Inflate tires to normal pressure (see "Detailed procedures for on-demand maintenance")</li> <li>✓ Repair tires</li> </ul>
2	The LEDs flash and the vehicle does not move	<ul style="list-style-type: none"> <li>✓ (indicated by error code).</li> </ul>	<ul style="list-style-type: none"> <li>✓ Call technical support.</li> </ul>
4	Handbrake is fully applied, but the vehicle does not stop	<ul style="list-style-type: none"> <li>✓ Loose handbrake mechanism</li> </ul>	<ul style="list-style-type: none"> <li>✓ The brake cable must be tightened firmly. Only by authorized technician</li> </ul>
5	When the drive lever is released on a flat surface, the <b>AFISCOOTER C</b> only comes to a standstill after more than 3 meters.	<ul style="list-style-type: none"> <li>✓ Control unit is faulty or incorrectly set</li> </ul>	<ul style="list-style-type: none"> <li>✓ Call technical support.</li> </ul>
6	The key switch is switched on, but the ON light is not illuminated.	<ul style="list-style-type: none"> <li>✓ Main fuse 50A tripped or electrical fault (See 2.1 in-Fig. 1)</li> </ul>	<ul style="list-style-type: none"> <li>✓ Reset fuse <b>Note:</b> Do not reset the fuse more than twice. If the symptom persists, contact your authorized dealer for assistance.</li> </ul>
7	The <b>AFISCOOTER C</b> does not move when you operate the drive lever.	<ul style="list-style-type: none"> <li>✓ Key switch is set to OFF</li> <li>✓ Charger is connected to the charging socket</li> <li>✓ The EMB is disengaged by the release lever</li> <li>✓ Other lock: Emergency brake</li> <li>✓ Disruption of the control system</li> </ul>	<ul style="list-style-type: none"> <li>✓ Set key switch to ON</li> <li>✓ Disconnect charger</li> <li>✓ Engage EMB lever</li> <li>✓ Switching the key switch off and on again</li> <li>✓ Call technical support.</li> </ul>
8	Irregular driving	<ul style="list-style-type: none"> <li>✓ Weak battery</li> <li>✓ Faulty connections</li> <li>✓ Defective battery</li> </ul>	<ul style="list-style-type: none"> <li>✓ Charge batteries for at least 18 hours</li> <li>✓ Remove the battery cover and check the battery connections. If the terminals are corroded, clean them with a damp, clean cloth</li> <li>✓ If the red LEDs remain lit, contact the dealer for a replacement battery.</li> </ul>
9	Main or rear lights or indicators are not functional.	<ul style="list-style-type: none"> <li>✓ Defective LED lighting board or electronic fault</li> </ul>	<ul style="list-style-type: none"> <li>✓ Call a technician for help</li> </ul>

No.	Symptom	Probable causes	Remedy
10	If you are driving uphill or carrying heavy loads, the speed will gradually reduce (but the scooter will move faster). (still).	✓ Overheating of the control unit	✓ Do not continue driving and allow the control unit to cool down

## Control unit error codes on the display:

The display can show the error code of the control unit in two different ways, which are defined via parameter 8 in the display settings:

### 1. Error code is shown on the alphanumeric display:

ERR01 - Battery connection faulty ERR02 -

Motor connection faulty

ERR03 - Motor has a short circuit to a battery connection ERR04 -

Freewheel was activated while driving

ERR06 - Electromagnetic brake was opened manually or panic button was pressed.

ERR07 - Drive control malfunction

ERR08 - control unit error - check that all plugs are secured ERR09 - faulty connection between control unit and electromagnetic brake

ERR10 - the voltage at the control unit is too high; check battery connections

### 2. Error code displayed as a series of flashing signals on the error display:

1 indicator	The battery must be charged or the battery connections are not in the correct position. Order. Check the connections on the battery. If they are OK, try charging the battery.
2 Flashes	The motor connection is not correct. Check all Connections between motor and control unit.
3 Flashes	There is a short circuit between the motor and a battery connection. Contact your workshop.
4 Flashes	The freewheel switch has been activated or the manual Brake disengagement mechanism has been actuated Check the position of the switch or lever.
5 Flashes	Not used
6 Flashes	The S drive is locked. Lock 2 is active. This may be because the battery charger is connected or because the seat is not in the driving position.
7 Flashes	Indicates a fault with the drive lever. Make sure that the The drive lever must be in the idle position before you switch on the scooter.
8 Flashing signal	Indicates a fault in the control unit. Check all Check connections for proper attachment.
9 Flashing signal	The parking brake connection is not correct. Check the connections between the parking brake and the motor. Make sure, that the connections of the control unit are secure.
10 Flashes	The voltage at the control unit is too high. Normally This is caused by a faulty battery connection. Check the battery connections.

## CHAPTER 11 - WARNING OF ELECTROMAGNETIC INTERFERENCE

All types of electric vehicles, such as electric wheelchairs and scooters (all types are referred to as "electric vehicles" in this text), can be susceptible to electromagnetic interference (EMI). These come from sources such as radio and television transmitters, amateur radio (HAM) transmitters, two-way radios and cell phones. The interference (from radio wave sources) can cause the electric vehicle to release its brakes, move on its own or drive in an unintended direction. It can also damage the electric vehicle's control system. The strength of the interfering electromagnetic energy can be measured in volts per meter (V/m). Every electric vehicle can withstand electromagnetic interference energy up to a certain level. This is referred to as its immunity level. The higher the immunity level, the greater the protection against electromagnetic interference.

At present, current technology is capable of achieving an immunity level that provides useful protection against the more common sources of radiated electromagnetic interference. This vehicle has an immunity level of 20 V/m as delivered, without further modifications.

There are a number of sources of relatively strong electromagnetic fields in the everyday environment. Some of these sources are obvious and easy to avoid. Others are not obvious and exposure is unavoidable. However, we believe that you can minimize the risk of electromagnetic interference by following the warnings listed below.

The sources of radiated electromagnetic interference can be roughly divided into three types:

1. Hand-held transceivers (transmitter-receiver) in which the antenna is mounted directly on the transmitter unit. Examples of these are CB radios, walkie-talkies, transceivers for security forces, fire and police, cell phones and other personal communication devices.

**NOTE:** Some cell phones and similar devices transmit signals while they are switched on, even when they are not in use.

2. Mobile transmitters and receivers (transceivers) with a medium range, such as those used in police cars, fire engines, ambulances and cabs, usually have an antenna mounted on the outside of the vehicle.

3. Transmitters and transceivers for long ranges such as commercial radio transmitters (radio and TV antenna masts) and amateur radio equipment (HAM).

**NOTE:** Other types of handheld devices, such as cordless phones, laptops, AM/FM radios, televisions, CD players, cassette players and small appliances such as electric shavers and hair dryers, do not, to our knowledge, cause electromagnetic interference problems in your electric vehicle.

### Electromagnetic interference (EMI) and electric vehicles

As the electromagnetic energy quickly increases the closer you get to the transmitting antenna, the electromagnetic field of hand-held radio wave sources (transceivers) is of particular importance. It is possible to inadvertently bring high levels of electromagnetic energy close to the electric vehicle's controls when using these devices. This can affect the movement and affect the brakes of the electric vehicle.

The warnings listed below are therefore recommended in order to avoid possible malfunctions of the electric vehicle's control system.

### **WARNINGS**

Electromagnetic interference from sources such as radio and television transmitters, amateur radio transmitters (HAM), two-way radios and cell phones can affect electric vehicles.

Following the warnings listed below should reduce the risk of unintentional release of the brakes or movement of the electric vehicle, **which can lead to serious injury:**

- 1) Do not operate hand-held transceivers (transmitter-receiver) such as CB radios or Do not switch on any personal communication devices such as cell phones while the electric vehicle is switched on;
- 2) Pay attention to nearby transmitters, such as radio or television transmitters, and try to avoid their proximity.
- 3) If unintentional movements occur or the brakes are released, switch off the electric vehicle **as soon as it is safe to do so.**

4) Be aware that adding accessories or components or modifying the electric vehicle may make it more susceptible to electromagnetic interference (as there is no easy way to assess their effect on the overall immunity level of the electric vehicle).

5) Please report any incidents of unintentional movement or brake release and note if there is a source of electromagnetic interference in the vicinity.

**Important information**

1) 20 volts per meter (V/m) is a generally achievable and useful immunity level against EMC (as of May 1994). The higher the immunity level, the greater the protection.

2) This product supplied to you has an immunity level of 20 V/m.

## Appendix A - Technical data

Type:AFISCOOTER C3/C4

Manufacturer:Kibbutz Afikim 15148,

Israel Payload: 150 kg (a 100 kg dummy according to ISO 7176-11 was used for the tests, supplemented by 50 kg weights distributed approximately according to the weight distribution of the dummy)

### (Table with SI units)

Information (ISO 7176-15)					
Standard designation	Min.	Max.	Standard designation	Min.	Max.
Overall length with leg rest 4-wheel version 3-wheel version	1366 mm 1300 mm		Seat angle		°3
Total width	665 mm		Seat depth	450 mm	
Length when folded Condition			Seat width	460 mm	
Height when folded Condition	805 mm		Height of the seat at the Leading edge	660 mm	
Total weight (unladen weight) 4-wheel version 3-wheel version	122 kg 102 kg		Back angle		
Weight of the heaviest part			Backrest height	520 mm	
Static stability downhill			Distance between the seat plate and Base plate	255 mm	
Static stability uphill	13°		Leg to seat angle	93°	
Lateral static stability	10°	14°	Distance armrest to seat	160 mm	
Energy consumption			Armrest depth	280 mm	
Dynamic stability uphill			Handlebar diameter	65 mm	
Max. obstacle height that can be overcome	25 mm	100 mm	Horizontal position of the Axis		
Maximum forward speed		15 km/h 12 km/h 10 km/h	Min. turning circle 4-wheel version - 3-wheel version –	340 cm 255 cm	
Min. braking distance from maximum speed	2500 mm (on straight route)		Min. track radius		
Pneumatic tires - All tires	4,10/3,50-6"		Charger for charging Dry/gel acid batteries	24 V DC, max. Charging current 8 A	
Front tire pressure: Rear tire pressure:		25 psi 25 psi			
Battery type, capacity Battery dimensions Charger	Lead acid/gel/AGM, leak-proof, 2x 12 V, max. 40 - 70 Ah Dimensions L=197 mm x W=168 mm x H=175 mm Charging current max. 8 A				
Additional equipment	Steering column for left-handers		Rear basket		

### (Table with British units of measurement)

Information (ISO 7176-15)					
Standard designation	Min.	Max.	Standard designation	Min.	Max.
Overall length with leg rest 4-wheel version 3-wheel version	53,7" 51,2"		Seat angle		°3
Total width	26,2"		Seat depth	17,7"	
Length when folded Condition			Seat width	18,1"	
Height in Folded state	31,7"		Height of the seat at the front edge	26"	
Total weight (unladen weight) 4-wheel version 3-wheel version	268 lb 224 lb		Back angle		
Weight of the heaviest Part			Backrest height	20,5"	
Static stability downhill			Distance between the seat plate and Base plate	10"	
Static stability uphill	13°		Leg to seat angle	93°	

..

Static stability sideways	10°	14°	Distance armrest to seat	6,3"	
---------------------------	-----	-----	--------------------------	------	--

Energy consumption			Armrest depth	11"	
Dynamic stability uphill			Handlebar diameter	2,5"	
Max. obstacle height that can be overcome	1"	4"	Horizontal position of the axis		
Maximum forward speed		10 mph 8 mph 6.7 mph	Min. turning circle 4-wheel version - 3-wheel version –	134" 100,4"	
Min. braking distance from maximum Speed	98,4" (on straight route)		Min. track radius		
Pneumatic tires - All tires	4,10/3,50-6"		Charger for charging Dry/gel acid batteries	24 V DC, max. Charging current 8 A	
Front tire pressure: Rear tire pressure:		25 psi 25 psi			
Battery type, capacity Battery dimensions Charger	Lead acid/gel/AGM, leak-proof, 2x 12 V, max. 40 - 70 Ah Dimensions L= 7.7" W= 6.6" H= 6.9" Charging current max. 8 A				
Additional equipment	Steering column for Left-handed		Rear basket		

**Notes:**

\* All specifications are subject to change without notice.

\*\* The maximum speed is limited in each country according to local traffic regulations.

\*\*\* The braking distance downhill can be significantly longer than on a straight road.

Date: July 2019

Issue: 02; Revision 01

Products: **AFISCOOTER C, 4- and 3-wheel version**

Part number of the operating instructions PRC4301

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<b>1 Flashes</b>	The battery needs charging or there is a bad connection to the battery. Check the connections to the battery. If the connections are good, try charging the battery.
<b>2 Flashes</b>	There is a bad connection to the motor. Check all connections between the motor and the controller.
<b>3 Flashes</b>	The motor has a short circuit to a battery connection. Contact your service agent.
<b>4 Flashes</b>	The freewheel switch is activated or the manual brake disengagement mechanism is operated. Check the position of the switch or lever.
<b>5 Flashes</b>	Not used.
<b>6 Flashes</b>	'The S-drive is being inhibited from driving. Inhibit 2 is active'. This may be because the battery charger is connected or the seat is not in the driving position.
<b>7 Flashes</b>	A throttle fault is indicated. Make sure that the throttle is in the rest position before switching on the scooter.
<b>8 Flashes</b>	A controller fault is indicated. Make sure that all connections are secure.
<b>9 Flashes</b>	The parking brakes have a bad connection. Check the parking brake and motor connections. Make sure the controller connections are secure.
<b>10 Flashes</b>	An excessive voltage has been applied to the controller. This is usually caused by a poor battery connection. Check the battery connections.

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