Afiscooter SE

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Kibbuz Afikim, 15148, Israel

Tel: 972-4-6754814; Fax: 972-4-6751456

WARNUNG

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Tel: 972-4-6754814; Fax: 972-4-6751456

WARNING

Read the user manual carefully before using the **AFISCOOTER SE**. If any part of this manual is unclear, please contact your dealer or one of our service centers. Read this manual carefully: Improper use may result in injury or damage!

WARNING

Electromagnetic interference (EMI) can lead to malfunctions in electric vehicles and endanger the safety of users.

For your own safety, you should read chapter 10 ("WARNING AGAINST ELECTROMAGNETIC FAILURES") before using the **AFISCOOTER**

Use **SE**.

The serial number and date of manufacture can be found on the front of the vehicle on a metal plate attached to the front light cover of the **AFISCOOTER SE**.

(See 4.6 in Figure 3 on page 19)

WARRANTY

AFIKIM ELECTRIC VEHICLES strives to constantly improve its products, their quality and reliability in order to ensure customer satisfaction and strengthen its market position.

The warranty for the **AFISCOOTER SE** is subject to the warranty conditions of the local dealer.

INFORMATION

Only authorized dealers of **AFIKIM ELECTRIC VEHICLES** are authorized to provide technical information about **AFISCOOTER SE**.

Further information not contained in this user manual, e.g. technical updates, safety warnings, etc., will be provided by your local dealer who is in contact with **AFIKIM ELECTRIC VEHICLES**.

If you find incorrect information, please contact your local dealer.

CE LABELING

The registration number of Afikim Electric Vehicles with the competent authority (United Kingdom) is: CA000292.

Our authorized representative with the competent authorities is: 3The

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CHAPTER 1 - OVERVIEW OF THE *Afiscooter SE* (MAIN FEATURES)

Congratulations on choosing the AFISCOOTER **SE** electric scooter. The **AFISCOOTER SE** is a scooter for older and disabled people, but can also be used as a recreational vehicle.

Classification of the scooter: Class C according to EN12184.

The **AFISCOOTER SE** is designed for a maximum occupant weight of 160 kg.

The **AFISCOOTER SE** is designed for use by older and disabled people. A certain minimum level of motor, visual and cognitive skills is required to use the scooter. It is recommended that you consult a doctor to assess your ability to use the scooter.

The **AFISCOOTER SE** is an indoor and outdoor scooter designed for operation under normal environmental conditions from very low (-30 °C) to very high temperatures (+45 °C).

The **AFISCOOTER SE** also works in light rain, but prolonged use in heavy rain or snow is not recommended.

The **AFISCOOTER SE** is easy to operate. The relatively wide wheels allow the rider to ride on unpaved roads and moderately rough terrain and overcome obstacles up to 5 cm high.

Please refer to Appendix A: Technical data (on page 30) for the tire pressure.

The **AFISCOOTER SE** can safely negotiate inclines of up to 9° (16%). Do not attempt to negotiate inclines and declines greater than the maximum permitted for the scooter.

The braking system gradually stops the vehicle within a short distance after the accelerator lever is released.

The **AFISCOOTER SE** is equipped with front and rear shock absorbers that ensure a comfortable and smooth ride, even when the vehicle is used on moderately rough roads. A standard handbrake is also available.

The seat is spacious and comfortable and can be rotated for easy entry into the vehicle (standard seats). It can also be adjusted forwards and backwards. The seat height can be adjusted by an authorized technician.

All controls are located on the handlebars and are easy to operate.

Under normal conditions, the **AFISCOOTER SE** can travel about 40 km before the batteries need to be recharged. The distance is reduced if the scooter is frequently used on sloping surfaces and uneven terrain or when climbing on sidewalks, kerbs, etc.

After reading the entire manual and before using the **AFISCOOTER SE**, carry out a cursory visual inspection for damage. If you have any questions or doubts, please contact your dealer.

The **AFISCOOTER SE** was developed to use only maintenance-free sealed batteries.

A maintenance manual is available for specialized technicians.

A digital version of the user manual for visually impaired people can be downloaded from our website www.afiscooters.com.

CHAPTER 2 - Afiscooter SAFETY SYMBOLS *IF*

WARNING! The initial configuration of the scooter and all procedures described in this manual must be carried out by an authorized supplier or technician.

The symbols below appear throughout this manual and on the scooter to indicate warnings and other important information. It is important that you read and understand them thoroughly.

WARNING! Indicates a potentially hazardous condition/situation. Failure to follow the specified procedures may result in injury, component damage or malfunction. The scooter bears the symbol of a black triangle.

OBLIGED! These measures must be carried out as specified. Failure to comply with the prescribed measures may result in injury and/or damage to the equipment. A white square on a dark background is shown on the scooter.

FORBIDDEN! These actions are prohibited and must not be carried out under any circumstances. Performing a prohibited action may result in injury and/or damage to the equipment. A black symbol with a crossed-out circle is shown on the scooter.

Observe the following safety instructions and follow them exactly. Further warnings and cautions can be found in this user manual. Be sure to read all chapters of the manual carefully before using the **AFISCOOTER SE**. We also recommend that you read the manual again at regular intervals to check the instructions.

Familiarize yourself with the **AFISCOOTER SE** and its operation before riding. Ride slowly (speed limiter set to MIN) until you have familiarized yourself with the functions, safety features, braking capacity and limits of the **AFISCOOTER SE**.

Watch out for pedestrians when riding the **AFISCOOTER SE**. Always ride at a reduced speed in busy areas. Only ride where it is permitted in compliance with all local laws and regulations. The **AFISCOOTER SE** is 72.5 cm wide. Make sure that you do not collide with people or objects in the vicinity of the vehicle.

Never remove the safety elements of the AFISCOOTER SE, such as mudguards, plastic covers or the emergency brake lever. The warning signs must not be removed under any circumstances. If one of the warning signs is removed, becomes illegible or is covered with paint, ask your dealer for a new sign.

Never put your hands, fingers or legs in moving parts or under one of the protective covers, as these cover moving parts and surfaces that become hot.

When making adjustments (e.g. folding down the seat, adjusting the armrest, moving the seat forwards or backwards, adjusting the distance between the handlebars) and when operating the gas pedal lever or the handbrake, ensure that your hands and fingers are not trapped between moving parts.

Never load the AFISCOOTER SE with a total weight of more than 160 kg, taking into account the sum of the user's weight and that of other loads, including those in the baskets.

The load in the rear basket must not exceed 8 kg. The **AFISCOOTER SE** is designed to carry ONE person on the single seat model. Do not use the scooter with a second person on board, even if it is a child.

Never inflate the tires of the **AFISCOOTER SE** to a higher pressure than recommended on page 30.

Do not use the **AFISCOOTER SE** on ladders. Never attempt to drive over obstacles whose

The **AFISCOOTER SE** must not be driven over obstacles with an angle of inclination greater than the maximum permissible limit and obstacles whose height is greater than specified in the technical data. When driving over an obstacle, always drive so that the **AFISCOOTER SE** is facing the obstacle.

If for any reason the **AFISCOOTER SE** does not stop when the throttle control lever is released or stops further than expected, immediately use the handbrake to stop the **AFISCOOTER SE**, turn the ignition key to OFF and contact your dealer.

Do not use the **AFISCOOTER SE** in pouring rain or extreme humidity or at temperatures below -30 °C or above 45 °C.

WARNING, surfaces can reach very high temperatures

The seat weighs 21 kg. Each battery weighs at least 14 kg. The power pack weighs at least 25 kg. Be careful when lifting the seat and the batteries.

Instructions for lifting the seat and batteries can be found in Chapter 8: Maintenance and Chapter 9: Troubleshooting and repairs. When lifting, use your knees and legs and never your back.

If the **AFISCOOTER SE** is involved in an accident and is damaged, have it inspected by a specialist before using it again.

Batteries contain lead and acid, which are very corrosive. The batteries must be handled and disposed of properly.

Do not use as a seat in motor vehicles.

When driving, make sure that the backrest is in an upright position.

Only use maintenance-free sealed lead-acid batteries.

WARNING! The **AFISCOOTER SE** must not be used by children. Do not allow children to play near the **AFISCOOTER SE** without an adult present while the batteries are being charged.

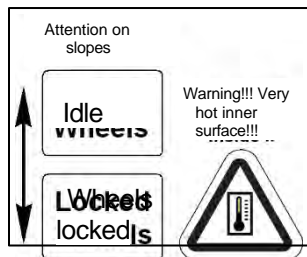
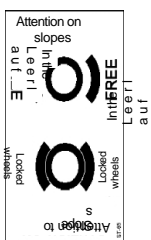
Drive carefully. Adjust the speed to the road conditions. If you increase the speed, the **AFISCOOTER SE** may become more unstable. Reduce the speed to a minimum before sharp bends to avoid the risk of tipping over.

The **AFISCOOTER SE is capable** of climbing slopes that exceed the recommended safety limits. However, avoid riding on slopes whose gradient you do not know. Ride very slowly and avoid riding on the edge of such slopes.

Before starting the engine, check that the emergency brake is working properly.

Do not release the manual parking brake or the hand lever of the electromagnetic brake when the **AFISCOOTER SE is** moving downwards. Never put the **AFISCOOTER SE in** neutral on sloping roads when you are sitting on or next to the scooter.

The **AFISCOOTER SE** lighting must always be switched on when visibility is restricted, both during the day and at night.

The seat cover is flame-retardant, so it must not be covered or replaced with another cover.

Watch out for overheated parts when removing the battery compartment cover. The motor, the axle, the electromagnetic brake and the electronic control unit can become hot. Do not touch them until you have checked the temperature.

Make sure that you do not get your fingers caught between the levers and the rubber stops.

Do not use cell phones, two-way radios, notebooks or other radio devices while using the **AFISCOOTER SE**.

Removing the earth connection can lead to electrical hazards. Only use 3-pin sockets and install a certified 2-pin adapter.

Do not connect any extension cables to the AC/DC transformer or the battery charger.

Keep tools and other metal objects away from the battery terminals. Contact with such tools can cause an electric shock. Do not sit on the **AFISCOOTER SE** when it is on a lifting device.

Danger of explosion

Disposal and recycling: The **AFISCOOTER SE** is made from recyclable materials (plastic parts, batteries, electronic components, etc.). For information on the correct disposal of the **AFISCOOTER SE**, the parts and the packaging of the scooter, please contact an authorized dealer.

Wear safety goggles.

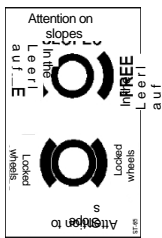
Contains lead.

The charger should only be used indoors.

Flammable material. Do not expose to naked flames.

The hand release lever for the electromagnetic brake is located on the rear of the **AFISCOOTER SE**. When operating this lever, make sure that you do not touch the motor surface, which can be very hot and cause injury. Observe the warning sign on the rear of the scooter near the lever.

CHAPTER 3 - SAFETY INSTRUCTIONS for the **AFISCOOTER SE**

GENERAL INFORMATION

OBLIGATORY! Do not use the **AFISCOOTER SE for the** first time without having fully read and understood the operating instructions.

The **AFISCOOTER SE is** a technologically advanced device that facilitates mobility and improves quality of life. Our company offers a wide range of products to meet all users' needs. La

The responsibility for choosing which model of **AFISCOOTER SE** to purchase and use lies with the user, who is capable of making this decision, and with the healthcare professional treating him/her (e.g. his/her doctor, physiotherapist, etc.).

The contents of this manual are based on the assumption that a person familiar with mobility aids has verified that the **AFISCOOTER SE is suitable for** the user and has assisted the prescribing physician and/or authorized dealer in the preparation of instructions for use of the product.

Under special circumstances, including certain medical conditions, the user must practice using the **AFISCOOTER SE** in the presence of a qualified person. The qualified operator may be a family member or a specialized professional who assists the user in performing daily activities.

When you start using the **AFISCOOTER SE** for everyday activities, there will probably be situations where the user will need to practice. Take the necessary time to practice how to control the vehicle safely, e.g. to go through doors, to go up and down an elevator or ramp and to drive over moderately uneven terrain.

Below you will find some recommendations, tips and other safety considerations that can help you to use the **AFISCOOTER SE** safely.

CHANGES

The **AFISCOOTER SE** was developed to offer the user maximum mobility and comfort. Under no circumstances is the user authorized to change, add, remove or deactivate functions, parts or components of the **AFISCOOTER SE**.

WARNING! Do not **make any** modifications to the **AFISCOOTER CSE** unless you have been authorized to do so by the manufacturer. Do not use any accessories that have not been tested or approved by the manufacturer.

REMOVABLE PARTS

WARNING! Do not attempt to lift or move the **AFISCOOTER by touching any of the removable parts, including armrests, seat or covers**.

SQUASHING POINTS

There are places on the **AFISCOOTER SE** where *there is a* risk of crushing. These are mainly moving parts such as wheels, throttle etc. Never put your hands, fingers or legs into moving parts.

SAFETY CHECK BEFORE THE JOURNEY

It is important to know the status and operation of the **AFISCOOTER SE** before using it. It is advisable to carry out a series of safety checks before riding to ensure that the **AFISCOOTER SE** is functioning correctly and safely.

Carry out the following checks before using the scooter:

- ✓ Check that the tire inflation pressure is correct. Maintain the pressure specified on each tire in psi/bar/kPa without exceeding it.
- ✓ Check all electrical connections. Ensure that they are secure and show no signs of corrosion.
- ✓ Check the connection of the cables. Make sure that the cables are firmly connected.
- ✓ Check the brakes.
- ✓ Check the charge status of the battery.
- ✓ If a problem is detected, contact an authorized **AFISCOOTER SE dealer** for assistance.

TIRE INFLATION

The **AFISCOOTER SE** is equipped with pneumatic tires. Check the tire pressure at least once a week. Maintaining the correct tire pressure increases the service life of the tires and contributes to the optimal operation of the **AFISCOOTER SE**.

AFISCOOTER SE.

WARNING! It is essential that the pressure in psi/bar/kPa specified on each tire is always maintained. Do not inflate the tires to a higher or lower pressure. Under-inflation can lead to loss of control of the scooter, while over-inflation can cause the tire to burst. If the specified tire pressure is not maintained, the tire and/or wheel may malfunction.

WARNING: Inflate the scooter tires using an air source equipped with a regulator and a pressure gauge.

NOTE: If the tires of the **AFISCOOTER SE** are only specified in psi, use the following conversion formulas to derive the corresponding pressure in bar or kPa:

$$\text{Bar} = \text{psi} \times 0.06895$$

$$\text{kPa} = \text{psi} \times 6.89476.$$

WEIGHT LIMITS

The scooter has a maximum load capacity. Further information can be found in Appendix A - Technical data.

OBLIGATORY! The load capacity specified for the **AFISCOOTER SE** must not be exceeded. Failure to observe this will invalidate the warranty.

Our

The company is not liable for injuries and/or damage to property caused by non-compliance with the weight limits.

WARNING! Do not carry people on the scooter. Carrying people on the **AFISCOOTER SE** can make it unstable.

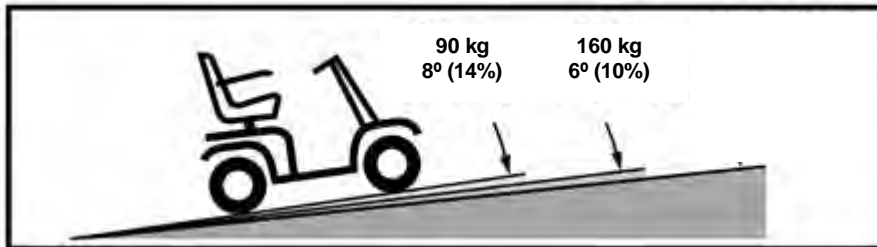
INFORMATION ABOUT SLOPES


More and more buildings are equipped with ramps with certain gradients that enable safe access. Some ramps may have very tight bends (180 degrees) that require a certain amount of skill when maneuvering the **AFISCOOTER SE**.

- ✓ Proceed with extreme caution when driving down a ramp or sloping surface.
- ✓ Make a wide arc with the **AFISCOOTER SE** to negotiate tight bends. This way, the rear wheels of the scooter make a wider arc. Do not cut a corner and do not get stuck in the corner of a railing.
- ✓ When descending a ramp, keep the **AFISCOOTER SE** at the minimum speed in order to master the descent safely.
- ✓ Avoid abrupt stops or starts.
- ✓ Refer to figure A to determine the maximum inclination, which depends on the speed of the scooter.

- ✓ Speed of the **AFISCOOTER SE** 15 km/h:

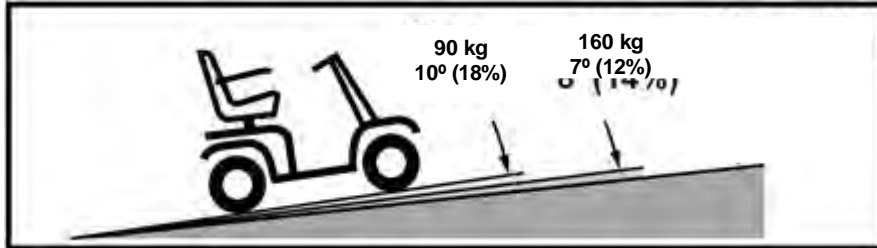- ✓ Speed of the **AFISCOOTER SE** 12 km/h:

- ✓ Speed of the **AFISCOOTER SE** 10 km/h:

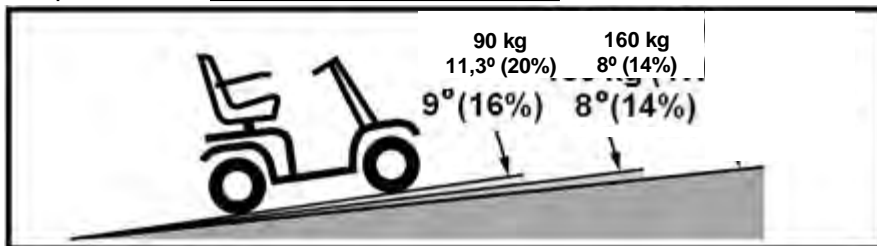
Figure A. Recommended maximum angle of inclination

When riding on a slope, try to keep the **AFISCOOTER SE** moving at **all times**. If it is necessary to stop, start slowly and then accelerate gradually. When driving downhill, set the speed switch to "Reduced speed" mode and only drive in forward gear. If the **AFISCOOTER SE** descends **the slope** at a higher speed than desired, stop completely by releasing the throttle control lever and then pushing the same lever slightly forward to make a controlled descent.

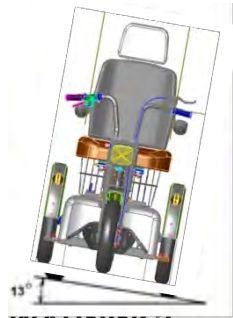
WARNING: Never put the **AFISCOOTER SE** in neutral on sloping roads while sitting on or next to the scooter.

WARNING! Do not zigzag or drive diagonally when driving up a slope.

Climb on and keep the **AFISCOOTER SE** on a straight path at all times. This considerably reduces the risk of tipping over or falling. In general, always drive with extreme caution when driving up a slope.

WARNING! Do not ride the **AFISCOOTER SE** uphill or downhill if you are riding diagonally or crosswise. If possible, only stop when you have finished riding downhill or uphill.

WARNING! Do not ride on potentially dangerous slopes (e.g. if the surface is covered with snow, ice, grass or wet leaves).

Please note that public access ramps are not regulated by standards in all countries and therefore may not comply with the standard gradient.

Other slopes may or may not be designed for use with a scooter. Figure A shows the stability of the **AFISCOOTER SE** and its ability to drive uphill with different loads and under controlled test conditions.

WARNING!

Even if the **AFISCOOTER SE** is capable of climbing gradients greater than those shown in Figure A, you should avoid exceeding these specifications and always observe the specifications given in this manual.

Otherwise, the **AFISCOOTER SE** may become unstable.

The tests were carried out with the **AFISCOOTER SE** seat in the highest and rearmost position. Use this information as a reference. The ability of the **AFISCOOTER SE** to ride uphill is influenced by the weight of the rider, the speed of the scooter, the angle at which the incline is reached and the configuration of the **AFISCOOTER SE**.

WARNING! Any attempt to drive up or down a slope greater than that shown in Figure A can make the **AFISCOOTER SE** unstable and cause it to tip over.

WARNING! Do not transport oxygen cylinders weighing more than 6.8 kg. Do not load the rear basket with more than 8 kg.

Figure B.
stable driving position

Normal driving position
Figure C. More

When driving on a sloping road, it is advisable to lean slightly forward. **See illustrations B and C.** This shifts the center of gravity of the rider and the **AFISCOOTER SE to the front**, which increases the stability of the **AFISCOOTER SE**.

NOTE: If you let go of the throttle control lever while driving up a ramp, the **AFISCOOTER SE** may slide back about one foot before the brake engages. If you let go of the throttle control lever while reversing, the **AFISCOOTER SE** may slide back about one foot before the brake engages.

NOTES ON DEALING WITH CURVES

If you take a bend at excessive speed, your scooter may tip over. Factors that affect the risk of rollover include the speed at which you take a bend, the steering angle (tighter or tighter turning radius), uneven road surfaces, sloping roads, riding from an area with low traction to an area with high traction (e.g. when changing from a grass surface to a paved surface, especially when turning at high speed) and sudden changes of direction. Driving around bends at high speed is not recommended. If you feel that your **AFISCOOTER SE** is in danger of tipping over when negotiating a bend, reduce the speed and steering angle (i.e. reduce the turning radius).

WARNING! When negotiating a bend with a tight turning radius, reduce the speed and ensure a stable center of gravity. If you use the **AFISCOOTER SE** at higher speeds, increase the turning radius. This considerably reduces the risk of tipping over or falling. In general, you should always use common sense when negotiating bends.

NOTES ON BRAKING

The **AFISCOOTER SE** is equipped with two powerful braking systems.

- ✓ Regenerative: This type of brake uses electricity to quickly decelerate the vehicle when the accelerator lever returns to the center/stop position.
- ✓ Parking brake: This brake is activated automatically if the regenerative brake brakes the vehicle almost to a standstill or if the system loses power for any reason.

OUTDOOR DRIVING SURFACES

- ✓ The **AFISCOOTER SE** is designed to ensure optimum stability under normal riding conditions on dry and level concrete or asphalt surfaces. However, there are occasions when the scooter is used on other surfaces. For this reason, the **AFISCOOTER SE is designed to** work well on hard ground, grass and gravel. The **AFISCOOTER SE** can be used safely on lawns and in parks.
- ✓ Reduce the speed of the **AFISCOOTER SE** when riding over uneven terrain and/or softer, yielding surfaces.
- ✓ Avoid areas with tall grass that could get caught in the moving gears.
- ✓ Avoid surfaces with loose gravel and sand.
- ✓ If you are not sure about a surface, avoid it.

PUBLIC ROADS

WARNING! Do not use the **AFISCOOTER SE** on public roads.

The AFISCOOTER SE can be difficult to see in traffic. Observe all applicable traffic regulations for pedestrians. Wait until the road is clear and then proceed with extreme caution.

FIXED OBSTACLES (STEPS, CURBS, ETC.)

WARNING! Do not drive near raised surfaces, unprotected projections and/or obstacles and unevenness such as kerbs, canopies, steps, etc.

WARNING! Do not attempt to climb or overcome an obstacle that is too high.

WARNING! Do not attempt to reverse on steps, kerbs or other obstacles. There is a risk of the **AFISCOOTER SE** tipping over.

WARNING! Ensure that the **AFISCOOTER SE** is perpendicular to the kerb if you need to get on or off the scooter.

WARNING! Do not attempt to climb on kerbs that are more than 5 cm high.

PRECAUTIONARY MEASURES IN UNFAVORABLE WEATHER CONDITIONS

Do not expose the **AFISCOOTER SE** to unfavorable weather conditions.

If you are caught out by rain, snow or extreme cold or heat while using the **AFISCOOTER SE**, seek shelter immediately. Dry the **AFISCOOTER SE** completely before parking, charging or using it again.

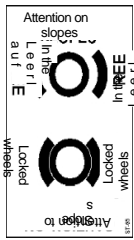
FORBIDDEN! Use in rain or snow, in salty air, in fog/humidity and on icy surfaces can lead to damage to the electrical system.

WARNING! If you expose the AFISCOOTER SE to extreme heat or cold for a long period of time, this can affect the temperature of the parts of the **AFISCOOTER SE** and in some cases cause skin irritation. Be careful when using the AFISCOOTER SE in extreme heat or cold or when the **AFISCOOTER SE is exposed to** direct sunlight for long periods of time.

WARNING The surface temperatures can rise if they are exposed to external heat sources such as sunlight.

NEUTRAL TRADE

The AFISCOOTER **SE is** equipped with a manual lever for neutral gear which, when pushed forward, allows the scooter to be pushed. Further information on engaging and disengaging neutral gear on the AFISCOOTER **SE can be found** in Chapter 4 - General description of the **AFISCOOTER SE**.

WARNING! When the **AFISCOOTER SE** is idling, the parking system is deactivated.

- ✓ Only switch off the drive motors when you are standing on level ground.
- ✓ Remove the key from the key switch.
- ✓ Stand next to the AFISCOOTER **SE** to engage or disengage the neutral gear. Never shift into neutral while sitting on the **AFISCOOTER SE**.
- ✓ After pushing the **AFISCOOTER SE**, always return to driving mode to lock the brakes (pull back the manual neutral lever).
- ✓ An exclusive function of the **AFISCOOTER SE** is a protection system called "**push-too-fast**", which prevents the scooter from reaching too high a speed when idling. The "**push-too-fast**" protection has two operating modes:
 - ✓ If the key is turned to the OFF position while the **AFISCOOTER SE is** idling, the **AFISCOOTER SE control unit** activates the regenerative brake if the scooter is driven at a higher speed than the programmed limit speed. In this case, the control unit controls the speed.
 - ✓ If the key is in the ON position while the **AFISCOOTER SE is** idling, you will encounter resistance regardless of the speed. This prevents the **AFISCOOTER SE** from reaching too high a speed if the neutral lever is accidentally raised while driving.
- ✓ When operating this lever, be careful not to touch the inside of the motor, which can be very hot and cause injury. Observe the warning sign on the back of the scooter near the lever.

STAIRS AND ESCALATORS

AFISCOOTER SE scooters are not suitable for use on stairs and escalators. Always use an elevator.

WARNING! Do not use the **AFISCOOTER SE** scooter to climb stairs or escalators.

DOORS

- ✓ Determine in which direction the door opens (towards you or in the opposite direction).
- ✓ Turn the rotary knob by hand or press the handle.
- ✓ If the door does not open towards you, drive the **AFISCOOTER SE** slowly forwards to push the door open.
When the door opens towards you, slowly guide the **AFISCOOTER SE** backwards to open the door.

ASCENSORS

Modern elevators are equipped with a safety mechanism at the threshold that reopens the elevator doors when activated.

- ✓ If you are standing on the threshold of an elevator when the doors start to close, press on the rubber part on the threshold or let the rubber touch the **AFISCOOTER SE** so that the doors open again.
- ✓ Make sure that bags, packages or accessories of the **AFISCOOTER SE** not get caught in the elevator doors.

NOTE: In some cases, it is difficult to maneuver the **AFISCOOTER SE** in elevators and at building entrances. Be careful when trying to maneuver the **AFISCOOTER SE** in tight spaces and avoid areas that could cause problems.

LIFTING DEVICES

If you are taking your **AFISCOOTER SE** on a trip, it may be necessary to use a lifting device to facilitate transportation. Please read the instructions carefully,

the manufacturer's specifications and safety instructions before using the lifting device.

WARNING: Do not sit on the **AFISCOOTER SE** when it is attached to a lifting device. The **AFISCOOTER SE** is not designed for such use and the user bears sole responsibility for any resulting damage or injury.

BATTERIES (see also Changing the batteries on page 28)

In addition to the warnings below, please also observe all other instructions for handling batteries.

REQUIRED! Terminals, clamps and similar battery accessories contain lead and lead compounds. Wear safety goggles and gloves when handling batteries and wash your hands afterwards.

WARNING! The scooter batteries are heavy (see Appendix A - Technical data). If you are unable to lift this weight yourself, ask someone to help you. Use suitable lifting techniques and avoid lifting weights that exceed your capabilities.

WARNING! Always store batteries before they freeze and never charge a frozen battery.

WARNING: Connect the battery cables correctly. RED (+) cables must be connected to the positive (+) battery terminals, BLACK (-) cables must be connected to the negative (-) battery terminals. Replace damaged cables immediately. Attach protective covers to all battery terminals.

NOTE: If a battery is damaged or has been damaged, immediately seal it in a plastic bag and contact your local disposal facility or an authorized **AFISCOOTER SE dealer** for recycling instructions.

TRANSPORTATION IN MOTOR VEHICLES

The manufacturer recommends that you do not sit on the **AFISCOOTER SE** while riding in a motor vehicle. The **AFISCOOTER SE** must be stowed in the trunk of a car or in the back of a truck or van after the batteries have been removed and the **AFISCOOTER SE** is securely fastened. In addition, all removable parts of the **AFISCOOTER SE**, including the armrests, seat and covers, must be removed and/or securely fastened during transportation.

WARNING: Although the **AFISCOOTER SE** is equipped with an optional seat belt, the belt is not suitable for securing the scooter when riding in a motor vehicle. When riding in a motor vehicle, each occupant must be secured with the seat belt on the vehicle seat.

WARNING: Do not sit on the seat while driving a motor vehicle.

AFISCOOTER SE.

WARNING: Ensure that the **AFISCOOTER SE** and the batteries are securely fastened during transportation. The batteries must be secured in an upright position with the covers on the terminals. Batteries must not be transported together with flammable or combustible materials.

BELT

Although the **AFISCOOTER SE** is equipped with an optional seat belt, the belt is not suitable for securing the scooter when riding in a motor vehicle. When riding in a motor vehicle, each occupant must be secured with the seat belt on the vehicle seat.

WARNING: Do not sit on the seat while driving a motor vehicle.
AFISCOOTER SE.

WARNING: Make sure that the **AFISCOOTER SE** and the batteries are securely fastened during transportation.

PREVENT THE SCOOTER FROM STARTING UP ACCIDENTALLY

WARNING! If you intend to stand still for a longer period of time, switch off the power. This will prevent the scooter from starting if you accidentally touch the throttle control lever.

GETTING IN AND OUT OF THE AFISCOOTER WHEN

Getting on and off the **AFISCOOTER SE** requires a good sense of balance. Observe the following safety tips when getting on and off the **AFISCOOTER SE**:

Remove the key from the key switch.

- ✓ Make sure that the **AFISCOOTER SE** is not idling.
- ✓ (see chapter 4 - General description of the **AFISCOOTER SE**).
- ✓ Make sure that the seat is locked in place.
- ✓ Fold up the armrests.

WARNING: Position yourself as far back as possible on the seat so that the **AFISCOOTER SE** does not tip over and nobody is injured.

WARNING! Do not place your entire weight on the armrests of the **AFISCOOTER SE** and do not load the armrests with weights, e.g. during transportation. This could cause the **AFISCOOTER SE** to tip over, which could result in the **AFISCOOTER SE** falling over and/or injuries.

WARNING: Do not place your entire weight on the ground. Otherwise there is a risk of the **AFISCOOTER SE** tipping over.

BEND AND STOOP

Avoid leaning or bending forward while riding the **AFISCOOTER SE**, as there is a risk of accidentally touching the throttle control lever. If you lean to the side while seated, there is a risk of tipping over. It is important to maintain a stable center of gravity to prevent the **AFISCOOTER SE from tipping over**. We recommend that you assess your personal limits and practise bending and tilting in the presence of a qualified assistant.

WARNING! Do not bend, stoop or lean forward to pick up objects from the floor of the **AFISCOOTER SE** or from one of the sides of the scooter. These movements could unbalance the center of gravity and weight distribution of the **AFISCOOTER SE** and cause the scooter to tip over.

FORBIDDEN! Keep your hands away from the tires and wheels while driving. Be careful if you are wearing loose clothing, as this could get caught in the tires and wheels.

DRUGS/PHYSICAL LIMITATIONS

The user of the **AFISCOOTER SE** must exercise caution and common sense when using the **AFISCOOTER SE**. For example, they must be aware of the safety risks when taking medication (prescribed by a doctor).

or over-the-counter) or if he has certain physical limitations.

WARNING: Consult your doctor if you are taking medication or have certain physical limitations. Certain medications and restrictions may prevent you from using the **AFISCOOTER SE** safely.

ALCOHOL/SMOKING

The user of the **AFISCOOTER SE** must exercise caution and common sense when using the **AFISCOOTER SE**. For example, they must be aware of the safety risks if they are under the influence of alcohol or smoke.

WARNING! Do not use the **AFISCOOTER SE** if you are under the influence of alcohol, as this may impair your ability to use the **AFISCOOTER SE**.

WARNING! The manufacturer strongly recommends that you do not smoke while sitting on the AFISCOOTER **SE**, even if the seat of the **AFISCOOTER SE** has passed the necessary test requirements for certification of ignition resistance in accordance with EN 1021-2:2006. Observe the following safety instructions if you decide to smoke cigarettes **while seated** on the AFISCOOTER SE.

- ✓ Do not leave a lit cigarette unattended.
- ✓ Keep ashtrays at a safe distance from the seat cushions.
- ✓ Make sure that the cigarettes are completely stubbed out before you throw them away.

ELECTROMAGNETIC AND HIGH-FREQUENCY INTERFERENCE

WARNING! Laboratory tests have shown that electromagnetic and radio frequency waves can impair the performance of electric mobility vehicles. Electromagnetic and high-frequency interference are e.g. e.g. from cell phones, mobile radios (such as walkie-talkies), radio repeaters, TV repeaters, wireless computer connections, signals to

microwaves, from paging transmitters and medium-range mobile transmitters and receivers used by emergency vehicles. In some cases, this interference can cause unintended movements or damage to the control system. Every electric vehicle has a certain immunity (resistance) to electromagnetic interference.

The higher the immunity level, the greater the protection against electromagnetic interference. This product has been tested and passed the tests with an immunity level of 20 V/M.

WARNING! Please note that cell phones, radios, laptops and other types of radio transmitters can cause unintended movements of the **AFISCOOTER SE** electric scooter due to electromagnetic interference. Be careful when using these devices while riding the **AFISCOOTER SE** and avoid standing too close to radio or TV repeaters.

WARNING! Adding accessories or components to a scooter may increase its susceptibility to electromagnetic interference. Do not make any modifications to the **AFISCOOTER SE** that have not been approved by the manufacturer.


WARNING! The **AFISCOOTER SE** can also interfere with other electrical devices in the vicinity, e.g. alarm systems.

NOTE: For more information on electromagnetic/radio frequency interference, see chapter 10 or contact an authorized dealer.


CHAPTER 4 - GENERAL DESCRIPTION OF THE Afiscooter SE

Note: The figures in brackets refer to the parts shown in Figures 1, 2 and 3.

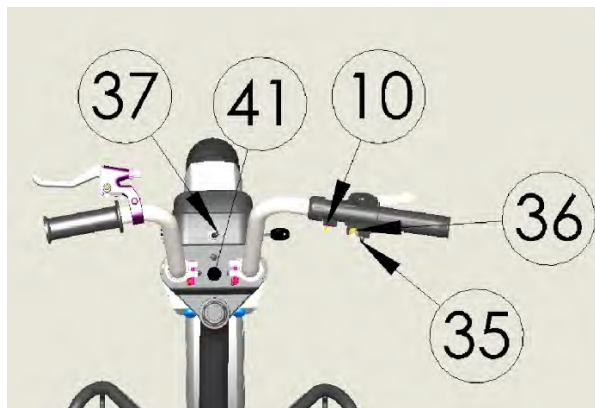Figure

1Figure 2

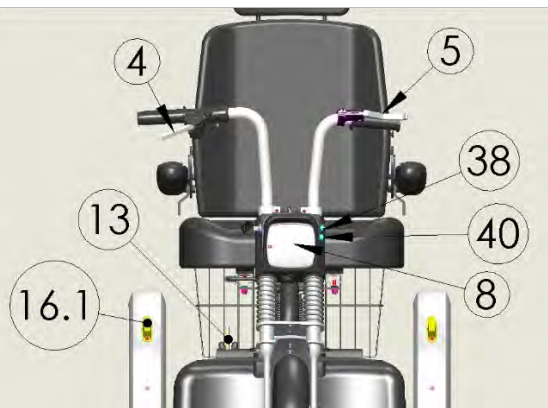
Figure

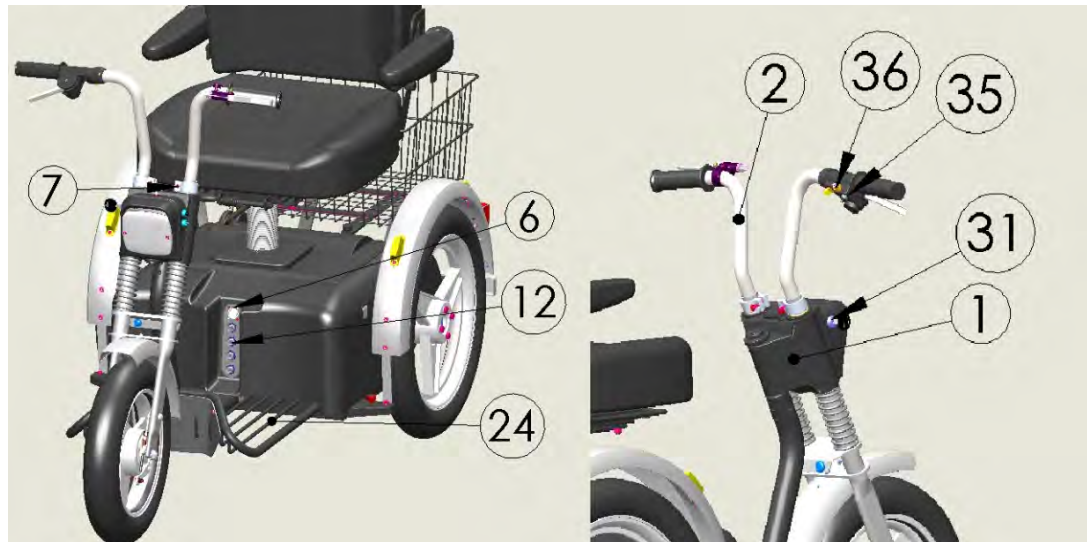3Figure 4

Figure

5Figure 6

N.	Illustration		
1	5	Nameplate with serial number and date of manufacture	The serial number and the date of the manufacture are indicated on a label on the luminaire cover.
2	5	Buttons	The AFISCOOTER SE can be used by both right-handed and left-handed users, but the factory configuration is equipped with right-handed controls. Left-handed controls can be configured by the dealer.
		Mirror	Every AFISCOOTER SE is equipped with 1 or 2 rear-view mirrors.
4	1	Throttle control lever	Used to control the speed of the AFISCOOTER SE .

5	1	Brake lever	Used for braking in emergency situations or as required.
6	6	Charging socket	Used to connect the charging plug to charge the battery.
7	6	Setting the buttons	The knobs can be adjusted up and down. Loosen the screw, make the adjustment and tighten it. screw again.
8	1	Headlight	Headlights for use in poor visibility
10	2	Switch for forward/reverse travel	
11	3	Cover of the battery compartment	Protects the power unit, electrical components and batteries.
12	6	Fuses	50 A main fuse, semi-automatic 10 A charging fuse, semi-automatic
13	1	Electromagnetic brake Hand lever	Manual release lever on the back of the AFISCOOTER SE ; you can push or pull the lever as indicated on the label:  = Press to release  = inserted (guideline). If the lever is in the unlocked position, it is not possible to lock the AFISCOOTER SE . use/drive. Note: After you have operated this lever, the system must be reset by first setting the key switch to OFF and then back to ON. Note: Never release this lever when the AFISCOOTER SE is moving downwards .
14	4	Rear bumper	
15	4	Rear light	
16	4	Flashing rear lights	
16.1	1	Front beacons	
17	3	Seat	The seat of the AFISCOOTER SE is equipped with folding armrests (7.3) to facilitate access on board. To make boarding easier the seat can be rotated
18	3	Backrest	
19	3	Headrest	

20	3	Armrests	They make it easier to get in. They can be folded forward the high
21	3	Seat lever forward/backward	For moving the seat forwards or backwards.
22	3	Lever for seat rotation	To turn the seat from 45° or 90° in both directions.
23	3	Adjusting the seat height	To adjust the seat height, please contact a technician approved.
24	6	Flatbed	Covered in plastic, which is cleaning requirements can be removed.

CHAPTER 5 - INSTRUCTIONS FOR SETTINGS

28	3	Sitting on the AFISCOOTER SE	<p>Before riding the AFISCOOTER SE, make sure that the plug is disconnected from the charging socket.</p> <p>Note: If the plug is inserted into the charging socket, the AFISCOOTER SE cannot be used.</p> <p>Use the seat swivel lever to turn the seat to one side and raise the armrest to make it easier to get in. Adjust the seat position using the seat forward/backward lever.</p>
----	---	-------------------------------------	--

CHAPTER 6 - INSTRUCTIONS FOR USE

6.1. GENERAL


31	5	Key switch	<ul style="list-style-type: none"> The ignition key switch is located on the right-hand side of the handlebars. <p>The key switch has two positions:</p> <ul style="list-style-type: none"> ✓ OFF - It is not possible to drive the AFISCOOTER SE and the electromagnetic brake is locked. ✓ ON - AFISCOOTER SE is switched on and the electromagnetic brake is automatically released when the throttle control lever is pressed. <p>To use the AFISCOOTER SE, insert the key into the key switch and turn it to ON (clockwise). The switch remains in this position and the green LED indicator lights up.</p> <p>Note: The key switch is used to switch the AFISCOOTER SE on, but not to lock it.</p>
----	---	------------	--

-		Automatic switch-off	If the key switch is in the ON position, the control system switches off automatically after a few minutes (usually 20) to conserve the battery. For automatic switch-off, turn the key switch to OFF and then back to ON.
33	3	Setting the handbrake	Contact an authorized technician to adjust the handbrake cable mechanism. The handbrake is part of the safety system. Do NOT adjust them yourself! Have them adjusted by a specialist.
34	1	Throttle control lever and manual handbrake	To drive: Press the throttle control lever lightly until the desired speed is reached. The speed is directly proportional to the inclination of the throttle control lever. To stop: Release the throttle control lever. The AFISCOOTER SE stops slowly and automatically. Note: The manual handbrake can also be used in emergencies.
35	2	Report	Before turning, press the corresponding signal switch, right or left To switch off the blinker, press the switch forwards. If a blinker is activated, the corresponding LED flashes. Depending on the model, an acoustic signal may also sound (if permitted by local regulations). To switch off the blinker, press the button
36	5	Acoustic signal buttons	Press one of the two buttons to activate the acoustic signal. The signal sounds automatically in the following situations (<i>except in cases prohibited by law</i>): 1. When reversing 2. When using the left or right blinker Note: It is possible to change the beep function by replacing a switch on the front panel. The procedure must be carried out by an authorized technician.
37	2	Light switch	Press the Light button to switch on the front and rear lights.
38	1	Button for the hazard warning lights	The emergency flasher button activates all the flashing lights of the AFISCOOTER SE . To switch off the hazard warning lights, press the button again. When the emergency flashers are activated, both LEDs flash.
-		Locking during the charging process	When the AFISCOOTER SE is being charged, lock mode is activated. It is not possible to ride the scooter in this mode. During the charging process, a signal lights up.
40	1	Safe guide/switch Reduced speed	Ride very slowly and carefully on ramps, high kerbs and tight bends and pay attention to the "Reduced speed" switch in "Reduced speed" mode.

			<p>Never attempt to drive over obstacles with an angle of inclination greater than the maximum permissible limit and obstacles with a height greater than specified in the technical data.</p> <p>Ride carefully and adapt your speed to the road conditions. Before sharp bends, reduce the speed of the <i>AFISCOOTER SE</i> using the "Reduced speed" setting to avoid the risk of Tipping over.</p>
41	2	Battery status display	Indicates the charge status of the battery
-		Parking	<p>After you have stopped the <i>AFISCOOTER SE</i>, turn the key switch to OFF and remove the key. The LED goes out.</p> <p>Note: The electromagnetic brake locks automatically when the throttle control lever is released and the <i>AFISCOOTER SE</i> is completely stationary.</p>

6.3 INSTRUCTIONS AND GENERAL USE

Overcoming an obstacle	To overcome an obstacle, drive the <i>AFISCOOTER SE</i> forwards towards the obstacle until the front wheel touches the obstacle, then increase the speed without changing direction.
Driving on slopes and in bends	<p>Reduce speed to a minimum before sharp bends, especially on sloping surfaces to avoid the risk of tipping over.</p> <p>The <i>AFISCOOTER SE</i> can cope with inclines and declines that exceed the recommended safety limits. Do not ride on slopes where you do not know the gradient. Always ride very slowly and avoid side slopes of more than 18% (10°).</p>
Before use	After reading the entire manual and before using the <i>AFISCOOTER SE</i> , carry out a visual inspection of all parts of the vehicle to ensure that they are not damage is present. If you have any questions or doubts, please contact your dealer.
Programming the control unit	Certain parameters of the <i>AFISCOOTER SE</i> control unit can be programmed. Programming may only be carried out by an authorized specialist. Incorrect programming can cause the <i>AFISCOOTER SE</i> to malfunction and lead to damage and injury in person.

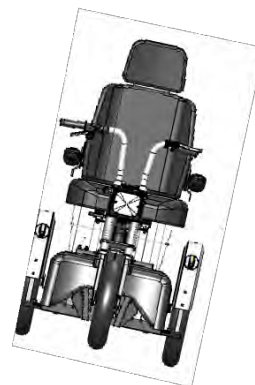Performance control	<p>It is strongly recommended that a functional check is carried out before using the AFISCOOTER SE for the first time or after a long period of non-use:</p> <ol style="list-style-type: none"> 1. Switch on the key switch. 2. Make sure that the green mains LED and the battery status indicator are both lit. 3. Slowly press the operating lever and check whether the AFISCOOTER SE moves at the speed corresponding to the degree of inclination of the lever. 4. Release the throttle control lever and check whether the AFISCOOTER SE gradually comes to a standstill and whether the "click" of the electromagnetic brake can be heard. 5. Check that all buttons on the handlebars are working properly. 6. Switch off the key switch. The AFISCOOTER SE is now ready for use.
Before use	<p>Before using the AFISCOOTER SE, make sure that you have checked your weight and that of the objects to be transported. The maximum total load is 160 kg.</p>
Safety belt	<p>Use the seat belt as a pelvic support or if required by the applicable traffic regulations. You can order a seat belt from your local supplier (see options for the AFISCOOTER SE in Appendix A).</p>
Fastening the AFISCOOTERs IF in a motor vehicle for transportation	<p>The AFISCOOTER SE must be secured for transportation in a motor vehicle (see Figure 8). Attach the 4 yellow straps (2 in front of and 2 behind the scooter) to the anchoring hooks of the vehicle using standard straps. Note: When transporting the AFISCOOTER SE, nobody may sit on the scooter.</p> <p style="text-align: center;">Figure 8</p>
Transportation	<p>To reduce the height of the AFISCOOTER SE during transportation, the seat can be removed as described in Chapter 8, Maintenance, and the handlebar column can be folded into a horizontal position.</p>

CHAPTER 7 - FEES

Read the charger manual before using it.

Note: If you are using a battery charger other than the one supplied with the AFISCOOTER SE, ask your dealer for instructions.

CHARGING THE BATTERY (see Figure 9)

1. Insert the charging plug (8) into the charging socket (10) located on the right-hand side of the handlebar column.
2. Plug the mains cable of the charger (9) into a socket (11).
3. When the charging process is complete, unplug the mains cable (9) of the charger from the socket (11) and then pull the plug (8) out of the charging socket (10).
4. Under ideal storage conditions, batteries that are charged to full capacity and are not used should be recharged every 6 months.
. If you do not intend to use the **AFISCOOTER SE** for a longer period of time, it is advisable to charge the batteries for two days and then disconnect them.
5. If the **AFISCOOTER SE** has not been used for a long period of time, charge the batteries for at least 24 hours before riding the scooter.

Recommended charging routine

1. Use the scooter during the day as required or until the charge level is low (see LCD charge level display).
2. Charge the scooter batteries at night at the end of the day.
3. It is not necessary to disconnect the batteries when charging is complete. The charger ends the charging process automatically.
4. It is not necessary to recharge the scooter's batteries after each ride unless you plan to use the scooter again before the end of the day.
5. If the scooter is to be charged during the day, carry out a full charge.
6. It is not recommended to leave the batteries empty for a few days in order not to affect their autonomy and service life.

- **If the charging plug is inserted into the charging socket, the electronic control unit of the AFISCOOTER SE interrupts the power supply to the electrical system and the scooter cannot be used.**
- **The charger supplied is suitable for charging dry lead/gel batteries. Only use the specified charger type. Contact your dealer before using a different charger to check compatibility.**

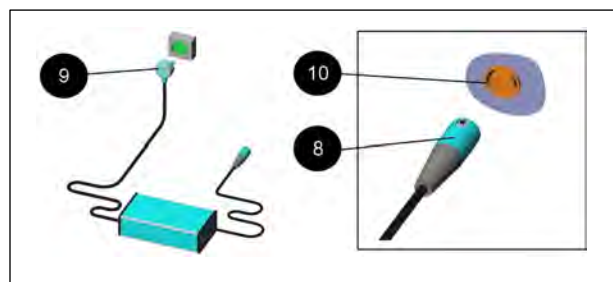
Figure 10

CHAPTER 8 - MAINTENANCE

Note:

1. A maintenance manual is available for specialized technicians.
2. The figures in brackets refer to the parts shown in Figure 11.

Although the **AFISCOOTER SE** requires only minimal maintenance, this should not be neglected.

The expected service life of the **AFISCOOTER SE** is 7 years, but can be extended considerably if the care and maintenance instructions are followed carefully.

When carrying out maintenance work on the **AFISCOOTER SE**, ensure that the scooter is standing on level ground and that the key is not inserted.

Regular maintenance procedures

✓ **Note:** No special tools are required for regular maintenance.

N.	Type of measure	Frequency
1	Check the pressure of all tires	Every week
2	Check the emergency brakes	Every week
3	Check that Run and Stop are functioning normally.	Every month
4	Check tire wear	Every month
5	Check whether any parts are missing or damaged (see figures 1 to 4).	Before using the AFISCOOTER SE for the first time or after a long period of non-use.
6	Check that all screws and parts are firmly tightened.	Once a year by a technician.
7	Replace batteries (to replace the batteries, the seat and the battery compartment cover under the seat must be removed).	<ul style="list-style-type: none"> • Every 2-3 years • After about 300-400 complete charging cycles /discharge cycles • If the range of the scooter is reduced on a single charge or • On the recommendation of a technical.
  Used batteries must not be disposed of with household waste. Information on the correct disposal of batteries can be obtained from an authorized dealer		
8	Cleaning	External surfaces: as required. Internal components: once a year by a technician.

Recommended periodic performance tests

Daily checks by the user

Throttle pedal: Switch off the scooter and check that the throttle mechanism is not bent or damaged and that it returns to its position when pressed and released. If there is a problem, cancel the safety checks and contact a service center.

Weekly checks by the user

Throttle lever: Move the throttle lever to the forward position for maximum speed and start the scooter. The scooter should not move. This indicates that the scooter has been switched on with the accelerator pedal already depressed.

If the scooter moves, contact a service center.

Parking brake: This test must be carried out on a level surface with at least one meter of free space around the scooter.

Switch the scooter on.

Check whether the status indicator remains lit or flashes slowly after half a second.

Drive the scooter slowly in forward gear until you hear the parking brake apply. The scooter can now start moving.

Release the accelerator pedal immediately. The parking brake should apply within a few seconds.

Repeat the test in the opposite direction. Cable and plug:

Check that all plugs on the scooter are securely plugged in and that the cables are intact.

Extraordinary maintenance procedures

Type of intervention	Frequency
Removing the seat and battery cover	Rotate the seat by turning the lever 45° (in either direction). The seat can only be removed at this angle. Removing the seat Hold the seat with one hand on the backrest and with the other hand on the front, lift the seat and pull it out of the seat. Assembly.
	Attention! The seat weighs 20 kg. The seat must be lifted by two people or with a suitable lifting device. Replace the battery compartment cover, put the seat back in place and make sure that it is positioned correctly. five screws.
Removing and refitting the compartment cover Batteries	To refit the battery cover, insert it back into the seat installation slot and make sure it is positioned correctly, then secure it with the four screws.
Batteries	Each battery weighs 23 kg. The batteries must be lifted by two people or with a suitable lifting device.
Tires	Inflating the tires to the correct pressure is important for riding and the stability of the scooter. Check the tire pressure every two weeks 25 ±2 psi for front and rear tires.
Cleaning	Only use a damp cloth and a mild cleaning agent for cleaning. Do not use rubber hoses as these can damage electrical and electronic components.
Replace batteries	Caution! Batteries should only be replaced by qualified personnel. Warning! Batteries contain a high energy source and can generate sparks that pose a fire hazard. Warning! Improper handling of batteries can lead to damage and injury. The procedure is shown in Figure 12. 1. Remove the insulating cap from one of the battery terminals. 2. Unscrew the cable from the terminal using a 10/13 mm wrench. Make sure that the wrench does not touch any other terminals or metal parts to avoid the risk of damage or injury. 3. Repeat steps 1 and 2 for all other terminals. 4. Remove the old batteries and insert the new ones. 5. Connect the cables of the new batteries as shown in Figure 11 (and on the label attached next to the batteries). 6. Screw the terminals tight and replace the insulating caps.

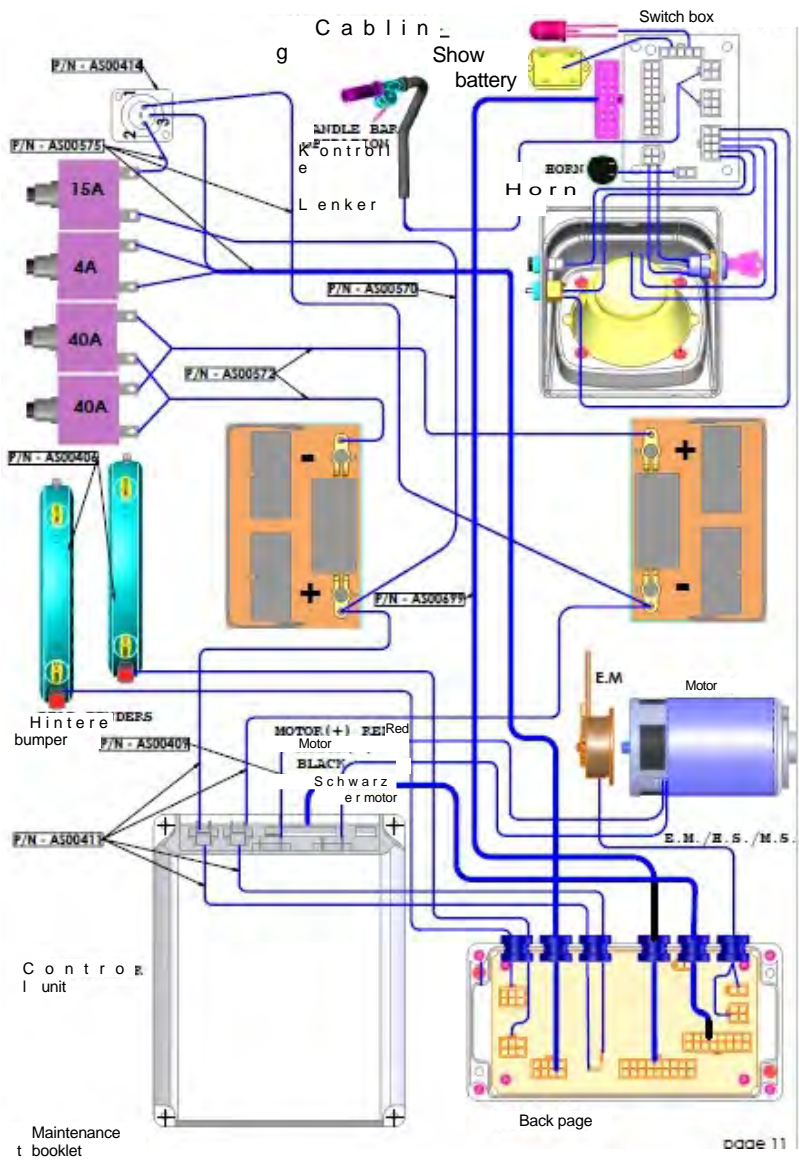
Figure 11

CHAPTER 9 - CONSERVATION

If the **Afiscooter SE** is to be stored **for a** longer period of time, e.g. longer than four months, we recommend storing it in a dry, clean and protected place, preferably where there is a socket for charging.

During storage, it is advisable to keep the **Afiscooter SE** covered at all times. If the scooter is not used regularly, charge the battery at least once a month until you resume normal use of the scooter. If the power switch has been left on or the scooter has not been charged for a long time,

It is not recommended to store the **Afiscooter SE** in extremely cold conditions. Storage at temperatures below 10 °C reduces the autonomy and capacity of the batteries.

WARNING: DO NOT FREEZE, DO NOT STORE BELOW 0 °C as this may lead to freezing. could lead to irreversible damage to the battery

CHAPTER 10 - TROUBLESHOOTING AND REPAIRS

The following table contains troubleshooting and repair instructions for some problems that may occur when using the **AFISCOOTER SE**, as well as suggestions for solving them. The table contains three columns:

Symptom	Probable cause	Solution
The problem that manifested	The possible causes, in the Order of probability (starting with the most likely probably)	The recommended Remedial measures for each Possible cause

For professional service, you can contact your dealer at any time.

N.	Symptom	Probable causes	Solution
1	Loss of tire pressure and unstable handling.	<ul style="list-style-type: none"> ✓ Low pressure ✓ Drilling 	<ul style="list-style-type: none"> ✓ Inflate the tire to normal pressure (see "Special maintenance procedures") ✓ Repairing the tire
2	The LEDs flash and the vehicle does not move.	✓ (indicated by the error code)	✓ Call technical support
4	The handbrake is applied, but the vehicle does not stop.	✓ The Handbrake mechanism has come loose	✓ The tension of the brake cable must be adjusted. The procedure must be carried out by a technician approved.
5	When the throttle control lever is released on level ground, the AFISCOOTER SE needs more than 3 meters to stop.	✓ The control unit is not working or needs to be adjusted	✓ Call technical support

N.	Symptom	Probable causes	Solution
6	The key switch is switched on, but the ON indicator does not light up.	<ul style="list-style-type: none"> ✓ Blown 50 A main fuse or electrical fault. (See 2.1 in Fig. 1) 	<ul style="list-style-type: none"> ✓ Replace the fuse <p>Note: Do not reinsert the fuse more than twice. If the symptom persists, contact an authorized dealer for assistance.</p>
7	The AFISCOOTER IF not moves when the throttle lever is pressed.	<ul style="list-style-type: none"> ✓ The key switch is switched off ✓ The charger is connected to the charging socket ✓ The lever to release the electromagnetic brake is disengaged. ✓ Other locking system: Emergency brake. ✓ Failure of the control system 	<ul style="list-style-type: none"> ✓ Switch on the key switch ✓ Disconnect the charger ✓ Lock the lever of the electromagnetic brake. ✓ Switch the key switch off and on again. ✓ Call technical support
8	Intermittent operation.	<ul style="list-style-type: none"> ✓ The state of charge of the battery is low ✓ Defective connections ✓ Defective battery 	<ul style="list-style-type: none"> ✓ Charging the batteries for at least 18 hours ✓ Remove the battery compartment cover and check the battery connections. If the terminals show signs of corrosion, clean them with a clean, damp cloth ✓ If the red LEDs remain lit, Contact your dealer to replace the battery.
9	The main or rear lights or blinker lights do not work	<ul style="list-style-type: none"> ✓ Defective LED light board or failure of the electronic components 	<ul style="list-style-type: none"> ✓ Call a technician for help.
10	When riding uphill or carrying heavy loads, the speed is gradually reduced. (but the scooter continues to drive).	<ul style="list-style-type: none"> ✓ Overheating of the control unit 	<ul style="list-style-type: none"> ✓ Stop and allow the control unit to cool down

The ECU error codes are shown on the display:

The ECU error codes shown on the display can appear in two different ways, depending on how parameter 8 of the display options has been set:

1. The error code is shown with an alphanumeric display:

ERR01 - Defective battery connection.

ERR02 - Defective connection to the motor.

ERR03 - Short circuit of a battery connection in the engine.

ERR04 - Idling activated while driving.

ERR06 - Electromagnetic brake opened manually or emergency button pressed.
ERR07 - Accelerator pedal error detected.

ERR08 - Error in the control unit - check that all connectors are correctly attached. ERR09

- Faulty connection between control unit and electromagnetic brake.

ERR10 - The voltage at the control unit is too high. Check the connections to the battery.

2. The error code is indicated by a series of flashing symbols on the error display:

1 Blink	The battery must be charged or one of the battery connections is defective. Check the connections on the battery. If the connections are OK, try charging the battery.
2 Flashing	One of the connections to the motor is defective. Check all connections between the motor and the control unit.
3 Flashing	A battery connection in the motor has a short circuit. Contact a service technician.
4 Flashing	The neutral switch has been actuated or the mechanism for releasing the handbrake has been used. Check the position of the switch or lever.
5 Flashing	Not used.
6 Flashing	S-Drive cannot be used for driving. Block 2 active". This could be because the charger is connected or the seat is not in the driving position.
7 Flashing	Accelerator pedal error displayed. Before switching on the scooter, check that the accelerator pedal is not jammed.
8 Flashing	Indicates an error in the control unit. Check that all connections are secure.
9 Flashing	Faulty connection to the parking brake. Check the connections on the parking brake and on the motor. Check whether the connections are secure.
10 Flashing	Excessive voltage at the control unit. This is usually caused by a loose battery connection. Check the battery connections.

CHAPTER 11 - WARNING OF ELECTROMAGNETIC INTERFERENCE

All types of electrically powered vehicles, such as electric wheelchairs and electric scooters (all referred to as "electric vehicles" in this text), may be subject to electromagnetic interference. Sources of this interference may include radio and TV repeaters, amateur radio transmitters, two-way radios and telephones

Cell phones. Interference (from radio wave sources) can release the brakes, cause movement or change the direction of an electric vehicle. It can also damage the vehicle's control system. The intensity of electromagnetic energy at the source of interference can be measured in volts per meter (V/m). Every electric vehicle is able to withstand electromagnetic interference up to a certain level. This property is referred to as the "immunity level". The higher the immunity level, the greater the protection against electromagnetic interference. Today's technologies are capable of achieving a level of immunity that provides sufficient protection against the most common sources of electromagnetic interference. This vehicle is supplied with an immunity level of 20 V/m without any further modifications.

There are numerous sources of relatively intense electromagnetic fields in any environment. Some of these sources are obvious and therefore easy to avoid. Others are less obvious and exposure is unavoidable. However, we believe that the risk of electromagnetic interference can be minimized if the following warnings are heeded.

Sources of radiated electromagnetic disturbances can be divided into three main categories:

1. Portable transceivers with an antenna attached directly to the transmitter unit. Examples include CB (Citizen Band) radios, walkie-talkies, public safety radios, fire and police radios, cell phones and other personal communication devices. **NOTE:** Some cell phones and similar devices transmit signals when they are turned on, even when they are not in use.
2. Mobile radios with medium range, e.g. in police vehicles, fire engines, ambulances and cabs. The antenna of these devices is usually mounted on the outside of the vehicle.
3. Long-range transmitters and transceivers, e.g. radio transmitters (radio and TV repeaters) and amateur radio equipment.

NOTE: To the best of our knowledge, other categories of portable devices, such as cordless phones, laptops, AM/FM radios, televisions, CD players, cassette players and small household appliances such as electric shavers and hair dryers, are unlikely to cause electromagnetic interference problems with an electric vehicle.

Electromagnetic interference and electric vehicles

As the electromagnetic energy rapidly increases in intensity as it approaches the transmitting antenna, the electromagnetic fields generated by portable radio wave sources (transceivers) are particularly dangerous. It is possible that high energy levels may be unintentionally generated when using these devices.

Electromagnetic emissions get close to the control system of an electric vehicle. This can affect the movement and braking of the vehicle.

It is therefore recommended that you observe the following warnings in order to avoid possible malfunctions of the electric vehicle's control system.

WARNINGS

Electromagnetic interference from sources such as radio and TV repeaters, amateur radio transmitters, two-way radios and cell phones can impair the operation of electric vehicles. If you observe the following warnings, you will reduce the likelihood of unintentional actions, such as releasing the brake or moving the vehicle, which **could result in serious injury:**

- 1) Do not use portable radios such as CB radios (Citizen Band) and do not switch on any personal communication devices, e.g. cell phones, while the electric vehicle is switched on.
- 2) Check whether there are transmitters such as radio or TV repeaters in the vicinity and avoid approaching them.
- 3) If the vehicle appears to move unintentionally or the brakes are released, switch off the vehicle, **as this is the safest measure.**
- 4) Note that the addition of accessories or components or modifications to the electric vehicle may increase its susceptibility to electromagnetic interference (there is no easy way to assess the impact on the overall immunity of the vehicle).
- 5) Report any accidents or unintentional braking movements/releases and check whether there is a source of electromagnetic interference in the vicinity.

Important information

- 1) 20 volts per meter (V/m) is an achievable level of immunity to electromagnetic interference in most cases (as of May 1994). The higher the immunity level, the greater the protection against interference.
- 2) When the customer purchases the product, it has an immunity level of 20 V/m.

Appendix A - Technical data

Type: AFISCOOTER SE Manufacturer: Kibbutz Afikim 15148, Israel

Maximum occupant mass: 160 kg (a 100 kg dummy according to ISO 7176-11 was used for the tests, to which a weight of 50 kg was added, distributed approximately proportionally to the weight distribution of the dummy)

(Table of metric dimensions)

Publication information (ISO 7176- 15)						
<i>Standard reference</i>	<i>min.</i>	<i>max.</i>	<i>Standard reference</i>	<i>min.</i>	<i>max.</i>	
Overall length with platform	1,480 mm		Angle of the seat level		3°	
Total width	725 mm		Effective seat depth	450 mm		
Folded length			Effective seat width	460 mm		
Folded height	670 mm		Height of the seat from the front edge	660 mm		
Total mass	146 kg		Angle of the backrest			
Mass of the heaviest part	21 kg (aggregate)		Height of the backrest	520 mm		
Static stability downhill			Distance between footrest and seat	255 mm		
Static stability in driveway	13°		Angle leg surface - seat	93°		
Lateral static stability	10°	14°	Distance between armrest and seat	160 mm		
Energy consumption	18AMP	100 AMP	Front position of the armrest structure	280 mm		
Dynamic Stability in driveway			Diameter of the knob			
Overcoming obstacles	25 mm	50 mm	Position of the horizontal axis			
Maximum speed in forward gear		12 km/h 10 km/h	Minimum turning radius	115 cm		
Minimum braking distance at maximum speed ability	2,500 mm (on rather flat surfaces)		Minimum turning radius			
Tires - All tires	Front - 3.00-10" Rear - 2.75-14"		Dry acid/gel charger	24 VDC charging maximum 8 amps		
Front tire pressure: Rear tire pressure:		20 psi 20 psi				
Battery type, capacity Battery size Charger	Lead-acid/gel/AGM sealed, 2x12 V 40-70 Ah max. Dimensions L=197 mm x W=168 mm x H=175 mm Charging current maximum 8 A					
Optional	Dumbbell for left-handers,		Rear basket			

(Table of imperial dimensions)

Publication information (ISO 7176-15)						
<i>Standard reference</i>	<i>min.</i>	<i>max.</i>	<i>Standard reference</i>	<i>min.</i>	<i>max.</i>	
Overall length with platform	58,2"		Angle of the seat level		3°	
Total width	28,5"		Effective seat depth	17,7"		
Folded length			Effective seat width	18,1"		
Folded height	26,3"		Height of the seat from the front edge	26"		
Total mass	322 lb		Angle of the backrest			
Mass of the heaviest part	46.3 lb		Height of the backrest	20,5"		
Static stability downhill			Distance between footrest and seat	10"		
Static stability when climbing	13°		Surface angle leg-seat	93°		
Lateral static stability	10°	14°	Distance between armrest and seat	6,3"		
Energy consumption			Front position of the armrest structure	11"		
Dynamic stability when climbing			Diameter of the knob	2,5"		
Overcoming obstacles	1"	1,92"	Position of the horizontal axis			
Maximum speed in forward gear		10 mph 8 mph 6.7 mph	Minimum turning radius	45"		
Minimum braking distance from maximum speed	98,4" (on accessible surfaces)		Minimum turning radius			
Tires - All tires	Front - 3.00-10" Rear - 2.75-14"		Battery charger for charging dry acid batteries /gel batteries	24 VDC Maximum fee 5 Traffic light		
Front tire pressure: Rear tire pressure:		20 psi 20 psi				
Battery type, Capacity Battery size Battery charger	Lead-acid/gel/AGM sealed, 2x12 V 40-70 Ah max. Dimensions L=7.7" x W=6.6" x H=6.9" Charging current maximum 8 amps					
Optional	Handlebars for left-handers,		Rear basket			

Notes:

* All specifications are subject to change without notice.

** The maximum speed is limited in accordance with the traffic regulations applicable in each country.

*** The braking distance on sloping surfaces can be significantly longer than on flat surfaces.

Date: DEC 2020

Edition: 01; Revision 8.1

Products: **AFISCOOTER SE, 3-wheeled vehicle**

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